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EN You can use this page to keep a record of your purchase and the contact details of the sales outlet together with your receipt.

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Website:

Content

Foreword	6
Technical specifications	7-8
Dimensions.....	8
Labels	9
EMC	10
Parts	11
General use	12-17
Product description.....	12
Target group.....	12
General warnings.....	12
Height adjustment.....	13
Tilt.....	13
Emergency stop.....	13
Emergency lowering.....	13
Display.....	14
Replacing the battery.....	14
Seat.....	15
Footrests and calf rest.....	15
Armrests.....	16
Head rest.....	16
Hip strap.....	17
To stand up from the chair.....	17
Maintenance	18
Maintenance.....	18
Cleaning.....	18
Disposal.....	18
Accessories	18

Foreword

Thank you for buying this Handicare product.

Your height adjustable tilting commode/shower chair comes from a complete line of chairs designed for both home use and use in healthcare institutions.

The commode/shower chairs supplied by Handicare are renowned for their stability, ergonomic design and durability.

Besides a complete line of wheeled and self-propelled wheelchairs, special commode/ shower chairs are also available for situations where a carer is present. These commode chairs can be tilted and/or adjusted in height, including by means of an electronic control for optimal ease of use by the carer. The chairs are very easy to use. For example, they can be wheeled completely over a toilet.

The height adjustable tilting commode/shower chair meets the needs of carers and users who expect a functional chair with many options. The chair height and posture can be adjusted continuously, enabling carers to always work ergonomically, without having to bend.

Please read these user instructions carefully!

Read all these instructions before you start using the chair. This will help you prevent unsafe situations and damage to the chair. The information in these user instructions is of vital importance to ensuring the proper use and maintenance of the chair. Unauthorised adjustments to the chair and/or accessories may negatively affect the safety of the chair. Handicare is not responsible for any accidents, incidents or failure of the product due to such unauthorised changes.

Definitions in the user instructions:

WARNING:

This means:

Safety warning. Any failure to heed this warning may result in personal injury.

CAUTION:

This means:

Important information. This instruction must be followed to enable this product to be used correctly.

Technical specifications

Height adjustable tilting commode/shower chair

Maximum user weight:	150 kg/330 lbs
Maximum pressure load per armrest:	75 kg/165 lbs
Dead weight:	51 kg/112 lbs
Width of undercarriage:	650 mm/25.6"
Length of undercarriage:	1065 mm/41.9"
Seat height adjustment range:	580-1040 mm/22.8"-40.9"
Tilting range:	0-35°
Turning radius:	Ø1135 mm/44.7"

Electrical components

Power supply:	24 V DC
Maximum power:	250 VA
Battery:	BAJ1 Linak
Battery charger:	CHJ2 Linak
Control box:	CBJ2 Linak
Safety category:	IPX5
Medical equipment:	Type B
Sound level:	45 dB(A); measured at no load (DS/EN ISO 3746)
Method of use:	ED max. 10%; max 2min ON, 18min OFF
Minimum/maximum temperatures:	Operation: +5°C to +40°C, Transport/ storage: -10°C to +50°C
Relative humidity:	Operation: 20% to 90% @ 30°C-not condensing, Transport/ storage: 20% to 90% @ 30°C-not condensing
Atmospheric pressure:	Operation: 700 to 1060 hPa, Transport/ storage: 700 to 1060 hPa


Recycling

Battery:	Cannot be recycled
Packaging:	Corrugated cardboard, can be recycled
Device:	Electrical, metal and plastic parts and components must be processed and recycled separately, according to the European Directive 2002/96/EC (WEEE)

Disposing of the height adjustable tilting commode/shower chair

This device is marked with the WEEE symbol according to the European directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In European countries, the WEEE-symbol, i.e. a wheelie bin with a cross through it, indicates that all electrical and electronic devices and batteries must be collected separately at the end of their service life. This requirement applies throughout the European Union. Do not dispose of these products as household waste. You can hand in your device and accessories at Handicare or your dealer, or you can contact your local authorities for information on how to dispose of the device.

Dimensions

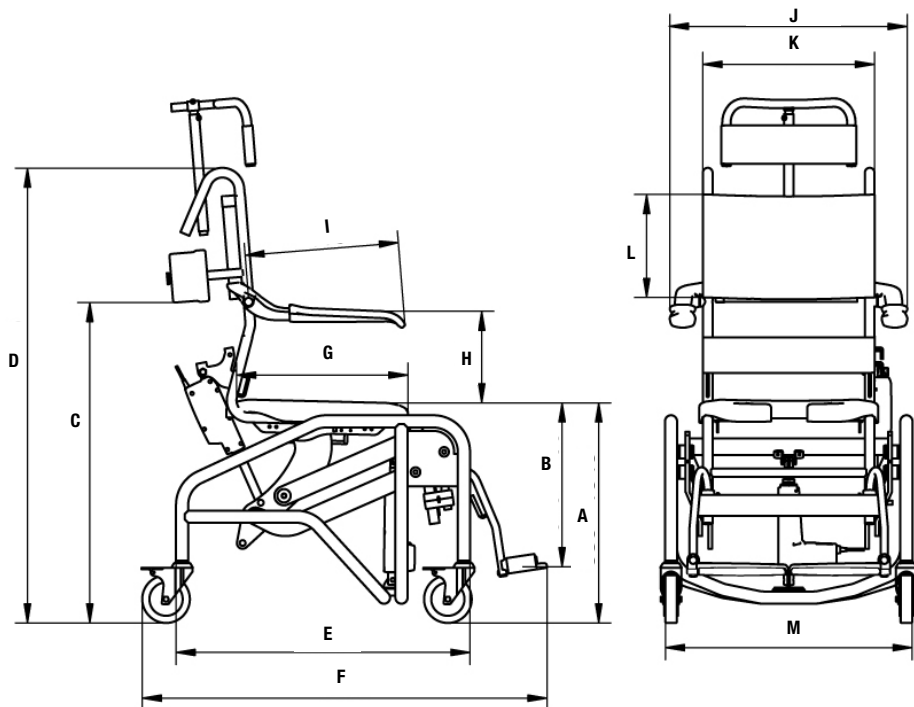	mm	inch
A	580-1040	22.8"-40.9"
B	375-455	14.8"-17.9"
C	840-1400	33.1"-55.1"
D	1195-1655	47"-65.2"
E	775	30.5"
F	1065	41.9"
G	450	17.7"
H	250	9.8"
I	410	16.1"
J	625	24.6"
K	450	17.7"
L	270	10.6"
M	650	25.6"

Labels

You can find the following information on the labels:

Key:

Electrical waste

Patient part of type B (Mand) as per EN/IEC 60601-1

CE Mark

Attention, consult accompanying document

IPXX

Protection against contact/foreign matter (first character) and water (second character) as per EN60529

MAX

Userweight


CSA-UL

TUV NRTL

CE label:

The CE label states the article number, production number and year of production, and manufacturer's details. It also states the maximum user weight.
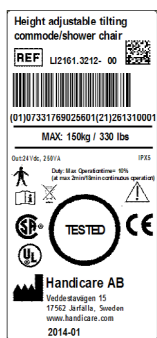


EMC

Guidance and manufacturer's declaration

Test	Specification	Comment
Radiated emission 10 m distance	IEC/CISPR 16-2-3:2006 Edition 2 EN 55016-2-3:2006	30 dB μ V/m, 30 - 230 MHz 37 dB μ V/m, 230 - 1000 MHz
RF-Electromagnetic fields	EN ISO 10535:2006 EN 60601-1-2:2007 clause 6	3 V/m, 80% AM, 1kHz 80 MHz - 800 MHz 10 V/m, 80% AM, 1 kHz 800 MHz - 2.0 GHz 3 V/m, 80% AM, 1kHz 2.0 GHz - 2.5 GHz.
Electrostatic Discharge	EN 60601-1-2:2007 clause 6 IEC 61000-4-2:2008 Edition 2 EN 61000-4-2:2009	\pm 6 kV contact discharge \pm 8 kV air discharge
Electrical fast Transients (Bursts)	EN 60601-1-2:2007 clause 6 IEC 61000-4-4:2004 + A1:2010 EN 61000-4-4:2004 + A1:2010	Signal and control lines: \pm 1 kV DC mains inputs and outputs: \pm 2 kV AC mains inputs and outputs: \pm 2 kV
Induced conducted disturbances	EN 60601-1-2:2007 clause 6 IEC 61000-4-6:2008 Edition 3 EN 61000-4-6:2009	3 Vrms, 80% AM, 1kHz 0.15 - 80 MHz

Parts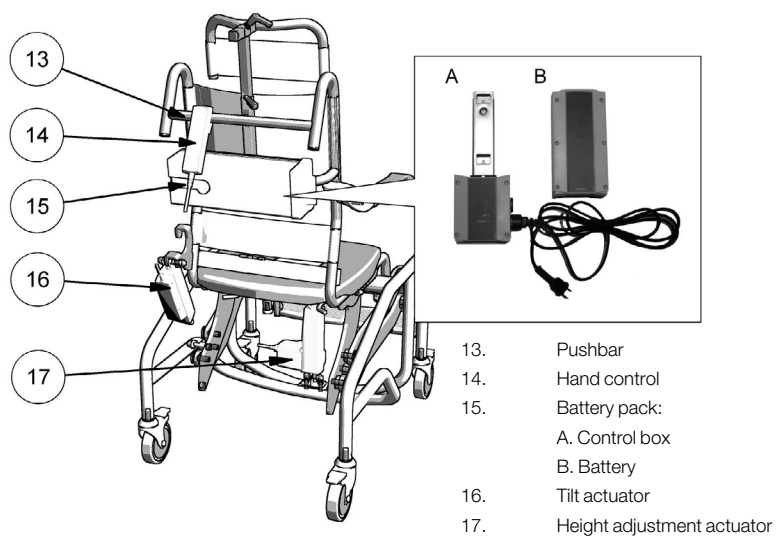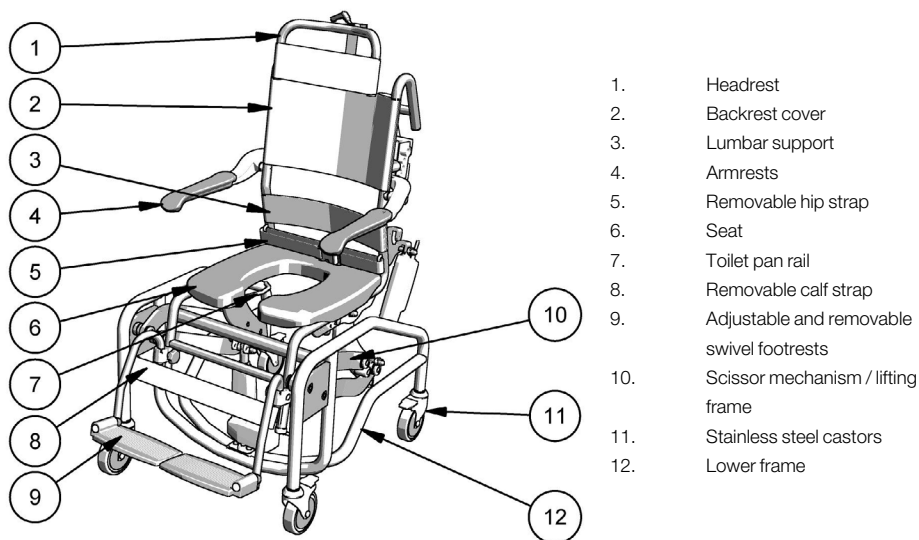

General use

Product description:

The height adjustable tilting commode/shower chair is intended to enable safe and comfortable showering, washing, drying and dressing. It can be wheeled over a toilet to use the toilet, but a bucket or a toilet pan can also be used for this.

The armrests serve various purposes, including offering support when sitting down and standing up; and they can be folded up individually to enable lateral transfers. The headrest also supports the head when the chair is tilted. The footrests support the feet, but they can be folded, swivelled away and removed for transfers. The seat can be removed to enable it to be reversed, cleaned and replaced. The height adjustable tilting commode/ shower chair can be adjusted to a comfortable working height for the carer, either electrically or hydraulically.

Target group:

Clients/People who need care and support while showering, going to the toilet and/or during other personal care activities. A seating facility offers a stable basic posture for the provision of care by third parties. The height adjustment and tilting functions of this chair enable carers to put their clients in the right posture in a way that is pleasant and ergonomically sound for both the client and the carer. This makes the care activities easier for both the client and the carer.

General warnings:


When not using the chair to transport a user, the castors should be rotated outwards and locked.

When folding down the armrest, do not hold the chair between the back of the armrest and the backrest of the chair as this might bring the risk of injury due to hands and fingers being trapped.

Only use the chair on a flat surface.

Only remove or relocate the toilet pan / bucket when the user is not in the chair.

Do not place the chair in direct sunlight as this might cause the surface of the seat or the backrest to become very hot.

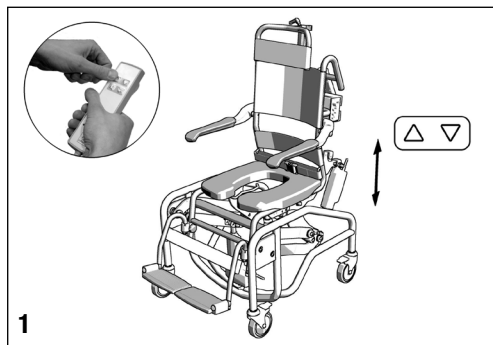
1

1. Height adjustment:

The height of the chair can be adjusted to set the right seat height so that carers can always work in an ergonomic posture. It also enables the height to be adjusted in order to wheel the chair over a toilet.

The height adjustment movement is driven by an actuator under the chair. The chair height can be adjusted using the upper hand control keys.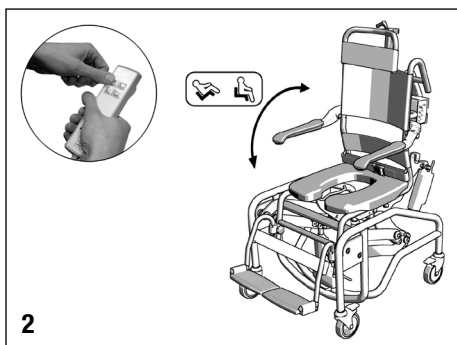

2


2. Tilt:

The tilt adjustment in the chair lets the carer tilt their client into a reclining position. This makes it easier to administer care to the client.

The tilting movement is driven by an actuator on the back of the chair.

The chair can be tilted using the lower hand control keys.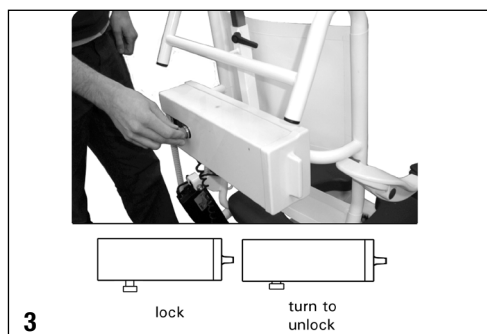


3

3. Emergency stop:

In an emergency: Press the red button to stop all functions (LOCK).

To reset this emergency stop turn the red button in a clockwise direction so that it comes out (UNLOCK).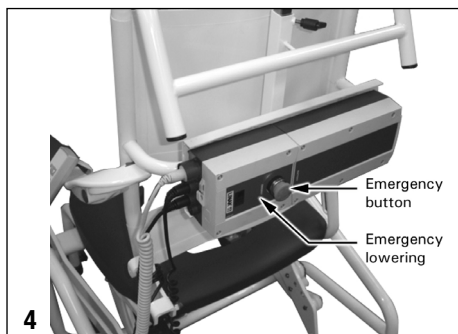


4

4. Emergency Lowering:

If the emergency button has been pressed, the Emergency Lowering function can be used to lower the chair.

The black Emergency Lowering button is located to the left of the emergency button on the control box.

Use a pointed object to depress the black button in order to activate Emergency Lowering.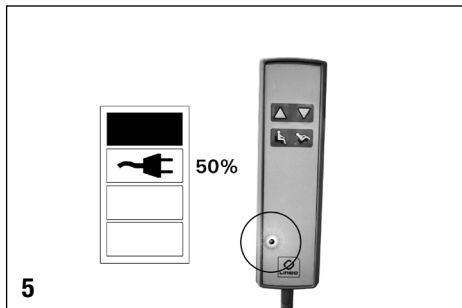


5. The display on the control box indicates the battery capacity.

When the remaining battery capacity is less than 50%, a plug symbol will be shown on the display and at the same time a LED will light up on the hand control.

Recharge the battery when these indicators show on the control box or the hand control.

Recharge the battery by removing it from the chair and placing it in the accompanying charger.

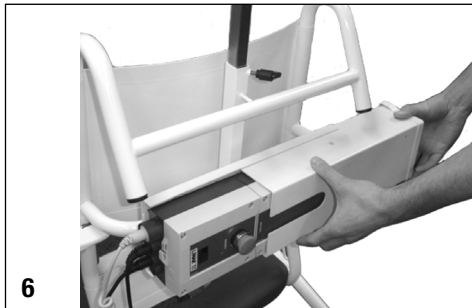6. Replacing batteries:

To remove the battery, first remove the protective cover. Slide the cover to the right until it comes loose from the frame.

7. Unlock the battery by squeezing the grip on the right-hand side. Tilt and at the same time slide the battery to the right to disconnect it.


⚠ CAUTION: given the weight of the battery, it should always be supported with two hands.

8. After removing the battery, it can be placed in the accompanying charger.

Since the control box remains fitted on the chair, a full battery may be installed immediately to continue using the chair.

Follow steps 6-8 in reverse order to install the battery.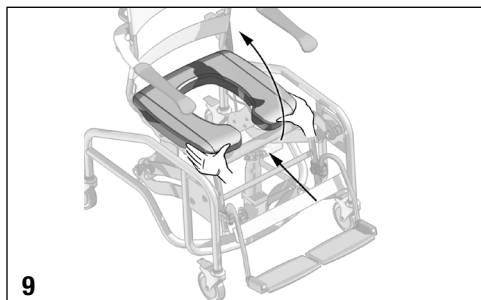

9


9. Seat:

The seat can be removed from the chair and placed on the chair in either of two directions.

Releasing the seat:

Unscrew the star knob (2x). Place both hands on the front of the seat and push the seat backwards in a straight line, all the way to the back of the chair.

Tilt the seat up while making sure it continues to be pressed in the back of the chair.

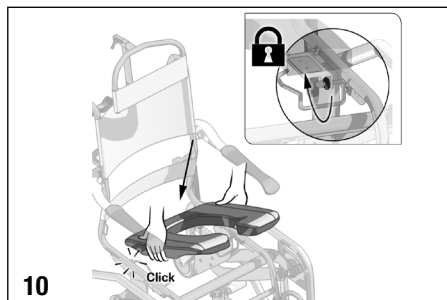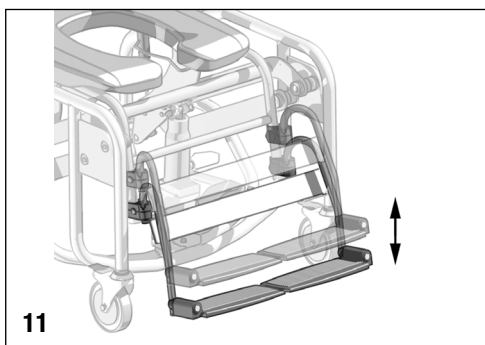
10

10. Installing and securing the seat:

Take hold of the seat with both hands and place it on the chair at a slanting angle.

Push and tilt the seat back and down until it clicks into position.

 Lock the seat by tightening the star knob (2x) on the thread under the seat.

11

11. Footrests and calf rest:

The footrests support the feet of the client while sitting. When the chair is tilted, the calf strap prevents the client's legs from getting under the chair. To properly support the client, the footrest and calf rest heights can be adjusted.

To adjust:

Unscrew the star knob (2x).

Slide the footrests to the desired height, making sure that both footrests are at the same height. Tighten the star knob (2x).

12

12. Folding up and swivelling away the footrests:

To facilitate a transfer, the footrests can be folded up and swivelled away. The footrest plate can be folded up independently of the footrests.

Remove the foot from the plate and fold the plate upwards.

To create extra space, remove the calf strap (fig. 13), lift up the tube in the mounting block and swivel the footrest outwards.

13. Removing the calf strap:


The calf strap can be removed quite easily by sliding it off the mounting pins in an upward movement.

14. Armrests:

The chair has foldable armrests. The armrests help the client to sit stably and offer extra support during transfers into and out of the chair. It is possible to fold up just one of the two armrests in order to create extra room for the carer, a lift or a lifting sling, or for lateral transfers.

⚠ WARNING:
Avoid your hands or fingers from getting trapped!
Never hold the backrest by its side when folding up the armrest.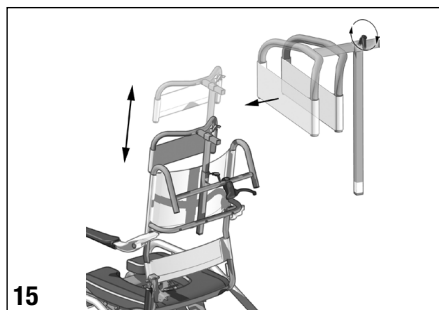

15. Headrest:


The headrest supports the head when the chair is in a tilted position. To ensure proper support, the headrest height and depth can be adjusted.

To adjust the height:

Release the height adjustment handle. Slide the centre of the headrest up or down to align it with the user's ear. Tighten the handle.

To adjust the depth:


Release the depth adjustment handle. Slide the headrest to the depth desired, Tighten the handle.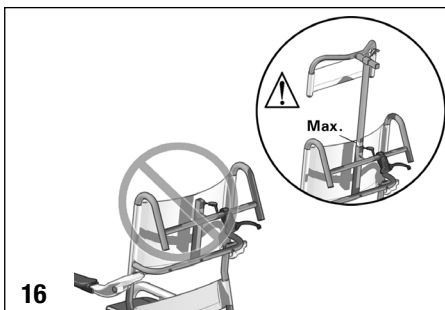

16. Headrest:

⚠ WARNING:
Prevent neck injury!
NEVER use the chair without the headrest.

⚠ WARNING:
Prevent neck injury!
Adjust the headrest within the maximum adjustment range!
This range is identified by the stainless steel / metal colour.
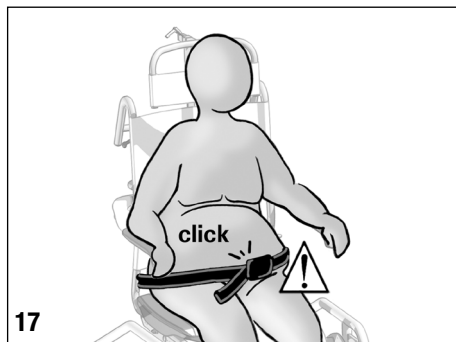
**17. Hip strap:**

The chair has a hip strap to ensure extra stability for the upper body during use.


To adjust the hip strap:

First adjust the strap so that it is wider than necessary for the client's body size.

Fasten the belt and then pull it to sufficiently tighten it around the client's body.

 **CAUTION:**


Check that the belt clasp is secured properly!

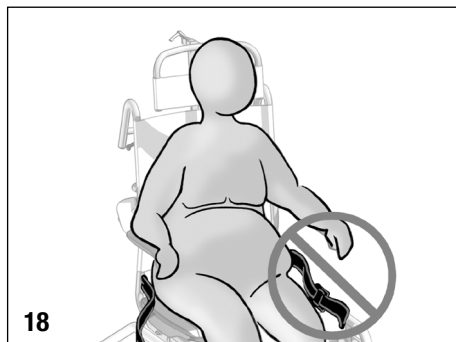**18****18. Hip strap:**

 **WARNING:**

Prevent injury!

ALWAYS apply the hip strap!

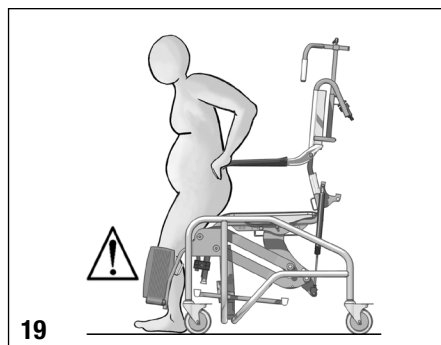
**19****19. To stand up from the chair:**

Adjust the chair to a height where the client can place their feet on the floor. Avoid that the client's knees have to be bent to an angle of less than 90 degrees.

Fold up the foot plates (fig. 12).

 **CAUTION:**

Make sure that the foot plates have been folded up completely and that there is sufficient space between the footrests before the client gets up from the chair!

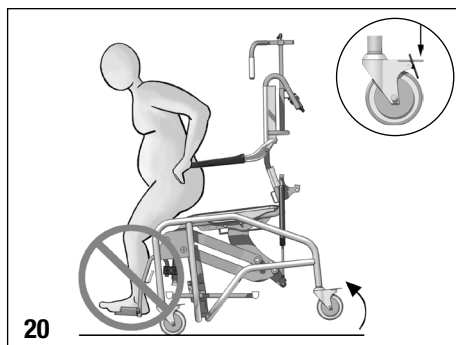**20****20. To stand up from the chair:**

 **WARNING:**

Prevent injury due to the chair tilting over!

NEVER stand on the footrests!

 **WARNING:**

Prevent injury due to the chair moving away while the client gets up!

ALWAYS rotate the castors outwards and apply the brakes (4x)!

Maintenance

Maintenance

To guarantee optimum chair operation the following checks and inspections must be carried out by the user on a monthly basis:

- Inspect the castors for dust and dirt and clean them.
- Inspect the brakes for dust and dirt and clean them.
- Check that the castors are fitted securely. If not, tighten the screws.

Make sure the chair is annually checked by a qualified service technician.

Please contact your dealer or Handicare for support immediately if the chair does not work properly.

Cleaning

Clean the chair after use.

Use domestic cleaning materials without abrasive or corrosive properties to clean the chair. Do not apply solvents and chlorine.

The chair can be disinfected using all disinfectants suitable for large surfaces and that have been approved by Health authorities.

The seat can be removed for cleaning.

Make sure to properly fit back all the parts that have been removed.

Disposal

Electrical, metal and plastic parts and components must be processed and recycled separately, according the European Directive 2002/96/EC (WEEE).

Accessories

Art.no.	Product name	Description
LI0121.917	Soft toilet seat	Toilet seat with reversible opening, to replace original seat
LI09506.636	Insert soft toilet seat	For toilet seat LI0121.917
LI0121.919	Toilet rails	Toilet rails, to replace original rails
LI02979.411	Bedpan	With lid and handle, stainless steel
LI02979.430	Bucket	With lid and handle, stainless steel

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Accessoires

LI0121.917	Soft toilet seat	Toilet seat with reversible opening, to replace original seat
LI09506.636	Insert soft toilet seat	For toilet seat LI0121.917
LI0121.919	Toilet rails	Toilet rails, to replace original rails
LI02979.411	Bedpan	With lid and handle, stainless steel
LI02979.430	Bucket	With lid and handle, stainless steel

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various power and manual wheelchairs, scooters, seating systems, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise- and recline chairs, and bathing and toilet-ing products.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se

Internet: www.handicare.com