

Technical mounting advice

For hinged and fixed arm supports, shower seats, floor consoles, height adjustment systems and swinging supports		
Type of wall	Suitable plugs	Suitable screws
Solid concrete, natural stone, solid red brick, sand-lime brick	Fischer Universalplug UX 10 art. 77869	Stainless steel hexagon head wood screws 8 mm
Solid/massive aerated concrete (Ytong, Durox), plaster block, pumice solid brick	Injection mortar + threaded rod. Possible mortars to use are: - FIS VS 150 C set art. 45302 or FIS VS 150 C art. 0509 - Fischer threaded rod M8	Stainless steel threaded rod M8
Perforated/hollow Hollow brick (Poriso), Hollow sand-lime brick	Injection mortar + anchor sleeve + screw inserts Possible mortars to use are: - FIS VS 150 C set art. 45302 or FIS VS 150 C art. 0509 - Fischer anchor sleeve FIS HK 14x70K art. 50436 - Fischer screw insert FIS E 8x85K art. 58050	Stainless steel hexagon head wood screws 8 mm
Plaster board, squeezed fibreboard, chipboard	Reinforce the wall, e.g. with plywood	Stainless steel hexagon head wood screws 8 mm
Concealed tank and carrier system, like Geberit, Wisa	Reinforce the wall, e.g. with solid water plywood, thickness 32 mm	Stainless steel hexagon head wood screws 8 mm

For grab rails, backrests, shower stretchers, washbasin rails and ceiling supports		
Type of wall	Suitable plugs	Suitable screws
Solid concrete, natural stone, solid red brick, sand-lime brick	Fischer Universal UX 8 art. 77869	Stainless steel hexagon head wood screws 6 mm
Solid/massive aerated concrete (Ytong, Durox), plaster block, pumice solid brick	Injection mortar + threaded rod. Possible mortars to use are: - FIS VS 150 C set, art. 45302 or FIS VS 150 C art. 0509 - Fischer threaded rod M6	Stainless steel threaded rod M6
Perforated/hollow Hollow brick (Poriso), Hollow sand-lime brick	Injection mortar + anchor sleeve + screw inserts Possible mortars to use are: - FIS VS 150 C set art. 45302 or FIS VS 150 C art. 0509 - Fischer anchor sleeve FIS HK 12x60K art. 50432 - Fischer screw insert FIS E 6x75K art. 58049	Stainless steel hexagon head wood screws 6 mm
Plaster board, squeezed fibreboard, chipboard	Reinforce the wall, e.g. with plywood	Stainless steel hexagon head wood screws 6 mm
Concealed tank and carrier system, like Geberit, Wisa	Reinforce the wall, e.g. with solid water plywood, thickness 32 mm	Stainless steel hexagon head wood screws 6 mm

! This mounting advice is a recommendation only. Suitable mounting methods and equipment must be determined at site and according to the type of wall. Handicare is not responsible for any inexpert mounting of products and the possible damage as a result of that.