# TurnSheet

SystemRoMedic™


- Manual - English
- Manual - Svenska
- Manual - Norsk
- Manual - Dansk
- käyttöohje - Suomi
- Gebrauchsanweisung - Deutsch
- Handleiding - Nederlands
- Manual - Français
- Manuale - Italiano
- Manual - Español

## REF

1592  
1588  
1589  
1597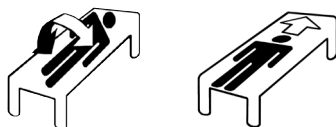

**SWL: 300 kg/660 lbs**

## Table of contents

<b>Always read the manual</b> .....	3
<b>English</b> .....	4-6
<b>Svenska</b> .....	7-9
<b>Norsk</b> .....	10-12
<b>Dansk</b> .....	13-15
<b>Suomi</b> .....	16-18
<b>Deutsch</b> .....	19-21
<b>Netherlands</b> .....	22-24
<b>Français</b> .....	25-27
<b>Italiano</b> .....	28-30
<b>Español</b> .....	31-33
<hr/>	
<b>Size and Accessories</b> .....	34
<b>Combination</b> .....	34

**EN Always read the manual**

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TurnSheet is bedded in the bed and features an anti-friction material that makes turning or repositioning higher up in bed easier for both the patient and the caregiver. TurnSheet consists of two sections: a mattress cover and a turning cover. The mattress cover is bedded around the mattress and the turning cover is then placed over it. The mattress cover consists of both anti-friction material and a friction surface.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.

## Functional inspection


---

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Functional inspection

Test TurnSheet's sliding function by rolling it and noting how easily it glides. If possible, compare with a new product. Similarly, test the friction material in the same way. Check to ensure that the surface is rough and still provides friction.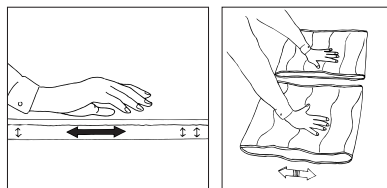

## Note


---

Be aware that there is a risk of the patient sliding out of bed. Take care to ensure that the mattress is sufficiently wide, so that there is anti-slip material on both edges of the mattress cover. Do not leave the patient unattended without first raising the bed rails.



Tuck the Slide panel under the mattress.

## Placement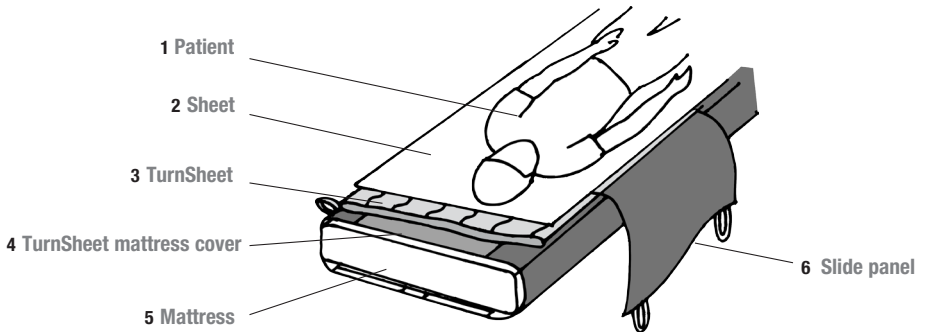

TurnSheet is placed and bedded in the bed in the order described above.  
 The mattress cover is secured around the mattress with Velcro strips.  
 The turning cover is placed on top, with the sliding surfaces facing each other.



## Turn manually with TurnSheet

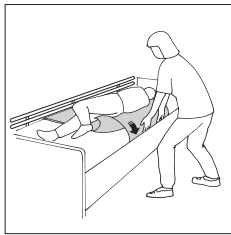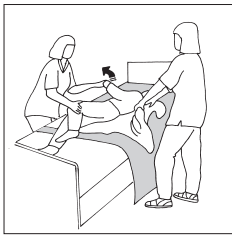Pull the patient to one side of the bed. Smooth out the sheet and turning cover. Pull on the sheet and turn the patient to the desired position.

Tuck the sheet under both sides of the mattress and raise the bed rails.

## Turning with TurnSheet, two parts (Art no. 1597)

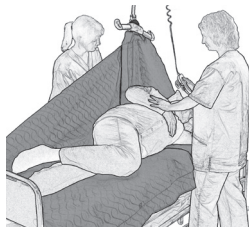
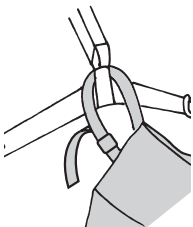
The TurnSheets suspension loop is hooked above the slingbar on a mobile or overhead lift.

If bed rails are not used, a caregiver should stand beside the bed to ensure safety.

Start turn by raising the slingbar. The caregiver with the best view of the patient operates the lift.

Turn the patient to the desired position.

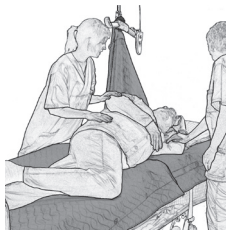
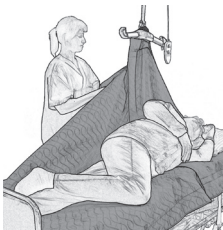


To ensure that the patient is repositioned in the centre of the bed, some manual adjustment towards the centre of the bed may be necessary during turning.

When the patient has been turned to the desired position, the Velcro strip can be loosened behind the patient.

Since the TurnSheet now supports the patient's back, the caregiver is free to attend to the patient's hygiene.

## Using slide panels

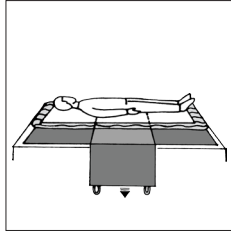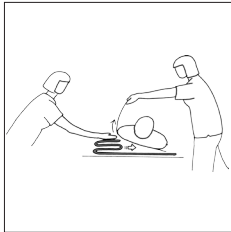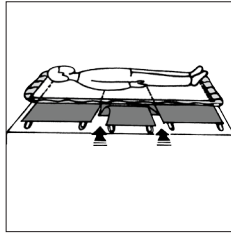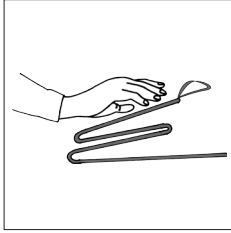Matress cover 140cm/ 55" has two slide panels and matress cover 200 cm/78" has three.

There is friction material under the slide panels.

The slide panels can be removed to enable positioning of the patient in bed. For example, the slide panel around the hips can be removed to prevent the patient from sliding downward when he/she attempts to sit up in bed.

Placing and removing the slide panels is easy, since the slide panel is folded against itself and slides in under the patient. Press down and pull forward. This is easier if one caregiver can tilt the patient somewhat.

When removing the Slide panel, tuck it under the mattress to prevent it from hanging down on the floor.

### Material:

Polyamid, polyester, microfibre

### Care of the product:

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

**CE** Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

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# SystemRoMedic™

For 25 years we have applied ourselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

**Transfer:** products for moving patients between locations.

**Positioning:** products for repositioning in the same location.

**Support:** products that provide mobility support.

**Lifting:** products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Our mission, to help people, has always been, and will continue to be, the driving force of innovation. We love easy transfers.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com)

**handicare**

Handicare AB  
Veddestav. 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)

Internet: [www.handicare.com](http://www.handicare.com)