English
Svenska
Norske
Danske
Deutsch
Français

Ready for life

Manual - ReadyGrip

Manual nr: 00819 Ver. 3 110211

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

English

Svenska

ReadyGrip is a very easy-to-use assistive device that facilitates raising from a recumbent position to a seated position. ReadyGrip is intended for active patients who have arm strength and good torso stability

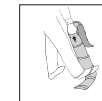
Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Pull on the handles to ensure that they withstand a heavy load. If there are signs of wear, the product must be discarded.

Always read the instructions

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

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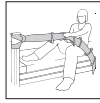
Placement of ReadyGrip

Fasten ReadyGrip by drawing the strap through the eyelet. Check to ensure that the fixture points will withstand the pressure of the load transferred by the straps.

English

Placement of ReadyGrip

Leave ReadyGrip in the bed, so that it is easily accessible for the patient (1).

The patient can now 'climb' up to a seated position with the help of ReadyGrip (2).

The patient can also use ReadyGrip as a support for sitting on the edge of the bed.

Material:

Polypropylene

Care of the product:

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

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