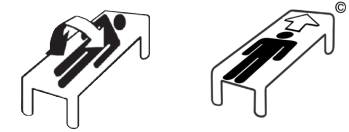
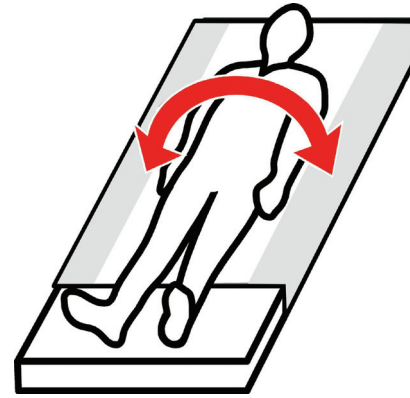
## Manual - ReadyLett

Manual nr: 741 - En, Nord Ver. 2 120920

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

Veddestav, 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299

Internet: [www.handicare.com](http://www.handicare.com)  
E-mail: [info@handicare.se](mailto:info@handicare.se)

### English

### Svenska

### Norske

### Danske

ReadyLett is a slide sheet that facilitates positioning and turning in bed. It is bedded in the bed like a normal sheet. ReadyLett is available in several sizes and models.

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### Functional inspection

#### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

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### Always read the instructions

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

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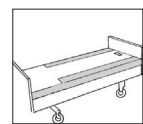
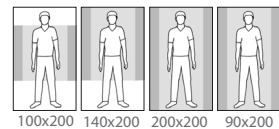
Do not leave the patient unattended during a transfer situation.

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

## Placement

Place ReadyLett with drawstring with the smooth surface in the middle of the bed. Ensure that the mattress cover's rough surfaces or evenly distributed on both sides of the bed. These rough surfaces prevent the patient from sliding out of bed.

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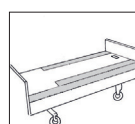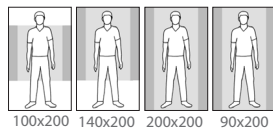
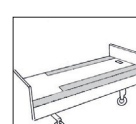
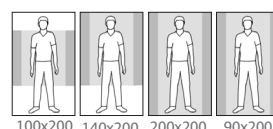
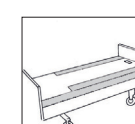
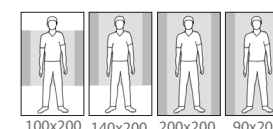
## Danske
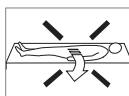
## Use

ReadyLett can be bedded in cases where the patient is able to turn himself/herself, but finds it easier with less friction.

**Note!** Where there is a risk of the patient sliding out of bed, use bed rails.

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
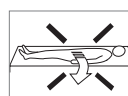
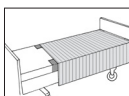
## Accessories

### SystemRoMedic


The transfer assistive products can be used either independently or as a complement to products in the SystemRoMedic range.

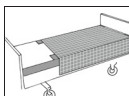WendyLett2Way and WendyLett4Way are two different draw sheets that can be used together with a slide sheet.

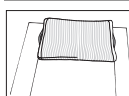WendyLett2Way is used for lateral repositioning and WendyLett4Way is used for both lateral repositioning and repositioning higher up or lower down in bed.

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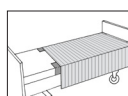
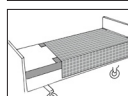
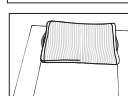
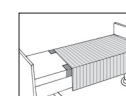
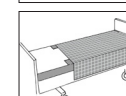
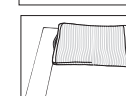
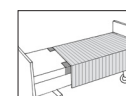
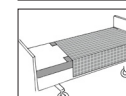
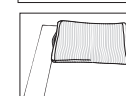
## Size:

11629	ReadyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	100 x 200/40" x 80"
11634	ReadyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	140 x 200/56" x 80"
11639	ReadyLett, 70 cm/28" sliding surface (bed 105 cm/42")	200 x 200/80" x 80"
11641	ReadyLett, 70 cm/28" sliding surface (bed 90 cm/36"), tailored with drawstring	90 x 200 /36" x 80"

## Material:

Polyester, cotton, satin

## Care of the product:

Read the product label.

Do not use rinsing agent. To prolong product life, avoid tumble-drying.

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