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# EN - Please read the instructions before you start using this product

## Intended use

This instruction applies to shower stool with soft and solid seats.

The shower stool is mostly used in the bathroom.

The shower stool is equipped with lateral armrests.

The soft seat is made of polyurethane; a soft material with good non-slip properties. The solid seat is made of ABS plastic. The shape of the solid seat also prevents slippage.

## Assembly – see drawing

The loose legs are mounted in the frame and secured with the snaplock clips. Make sure the clips are properly inserted in the leg holes. The seat is pressed onto the seat frame, with the 4 teats securely inserted in the 4 holes.

Check the correct position of the seat before use.

## Adjustment – see drawing

The shower stool is height adjustable in 8 steps. The adjustment takes place by detaching the snaplocks from the legs one by one and selecting the desired height.

Be sure to adjust the four legs to the same height and to insert the snaplocks firmly into the tubes again. The rear legs can be height adjusted to slope the seat forwards, however not more than 2 holes difference.

## Maintenance

To be wiped clean after use. Clean the stool with domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied.

The polyurethane seat can be detached for cleaning. Polyurethane can tolerate temperatures up to 85 °C/3 minutes. Be sure to reattach the seat correctly.

## Warning

- Max. load: 150 kg.
- For optimal safety, shower stools without a mounted backrest should be placed against a wall.
- Only use the shower stool as described in the paragraph "Purpose".
- Do not stand on the shower stool.
- Do not push or pull the shower stool across the floor while sitting on it!
- Check the correct position of the snaplocks before using the stool.
- Check the condition of the snaplocks frequently. Regularly check that the snaplocks are inserted correctly into the tubing.
- For optimal safety it is recommended to check the ferrules frequently and to replace them whenever necessary.
- Height adjustment of the rear legs should be max. 2 holes difference to the front legs.

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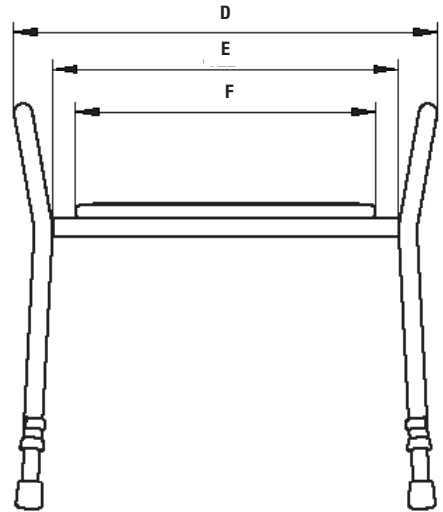
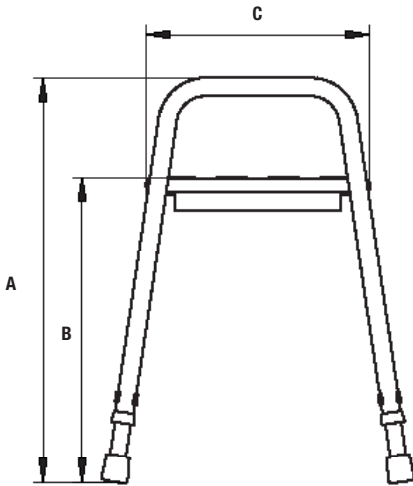
## **Always make sure that you have the correct version of the manual**

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## **For questions about the product and its use**

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå , SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)