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EN - Please read the instructions before using the product.

These instructions apply to the hip belt for Combi chairs.

Intended use

The hip belt fits all Combi chairs.

The hip belt is well suited for users who need trunk support, and can be adjusted to the individual user.

Installation

Remove the backrest by removing the clips on both sides (1). Lift up the backrest. Place the hip belt from the bottom up on to the backrest. Reinstall the backrest by placing it down into the holes in the frame. Put both clips back into the holes in the frame (1).

Adjustment and use

The hip belt can be adjusted to the individual user by loosening or tightening the strap on the front (2) or on the back of the hip belt (3).

Maintenance

The hip belt can be removed and machine washed at up to 30 degrees Celcius. It should not be dried in a tumbler dryer.

The hip belt can be cleaned with common cleaning solutions without abrasive or etching properties. It is not recommended to use solvents or chlorine.

Warning!

- Maximum weight of user: 100 kg.
- The hip belt must only be used as described under the section "Intended use".

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Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com