Toilet raiser with lid

User manual - English

Bruksanvisning - Svenska

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REF

10735

SWL: 130 kg/285 lbs

Table of contents

English	3
Svenska	4
Deutsch	5
Nederlands	6
Technical information	7

Toilet raiser with lid, raises toilet seat height by 100 mm/4"

Safety information

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use

Ensure that the product is correctly mounted/installed.
Be sure that adjustment knobs on sides of seat are tight.

Always read the user manual

Always read the user manuals for all assistive devices used. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Installation instructions

Remove toilet seat and lid from toilet. Turn both locking clamps counter clockwise until they can no longer turn. This will open the locking devices. Place raised toilet seat on and position in a secure fashion. Be sure the two locking brackets are on both sides of the wall and that the seat is flush with the. Turn the locking brackets clockwise until tight. Hand tighten only. Do not use tools as threads on the adjustment bracket may strip.

Removal instructions

Rotate knobs counter clockwise until seat is loose. Lift front of toilet raiser and pull forward to remove.

Using the product

The toilet raiser increases the height of a toilet by 100 mm/4". User's weight should be centered over toilet seat.

Warning

- Do not load the seat surface on one side only.
- Under no circumstance stand on the seat.

Material

Plastic

Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

Medical Device Class I

The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Verwachte levensduur product

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Technical information

Material	Plastic
Total depth	412 mm, 16.2"
Total height	155 mm, 6.1"
Raiser	100 mm, 4"
Hygiene cutouts	In front and back
Toilet opening, width x length	200 mm x 270 mm, 7.8" x 10.6"
Weight	1.5 kg, 3.3 lbs
Max. user weight	130 kg/285 lbs

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com