User manual – English

Bruksanvisning – Svenska

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
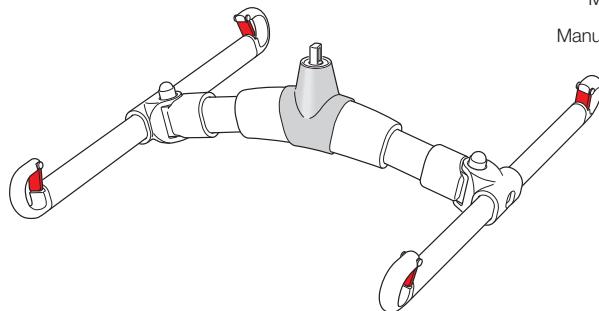
REF

70200072

**SWL: 454 kg / 1000 lbs**
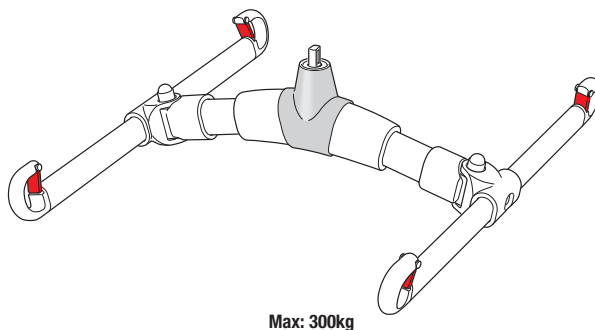
## Table of contents

<b>English</b> .....	<b>3-7</b>
<b>Svenska</b> .....	<b>8-12</b>
<b>Norsk</b> .....	<b>13-17</b>
<b>Danske</b> .....	<b>18-22</b>
<b>Suomi</b> .....	<b>23-27</b>
<b>Deutsch</b> .....	<b>28-32</b>
<b>Netherlands</b> .....	<b>33-37</b>
<b>Français</b> .....	<b>38-42</b>
<b>Italiano</b> .....	<b>43-47</b>
<b>Español</b> .....	<b>48-52</b>

BariBar is a four-point aluminium/steel sling bar which is used, in combination with a correctly fitted lifting sling and a ceiling lift unit approved by Handicare, for lifting of users weighing up to 454 kg/1000 lbs. BariBar opens up the lifting sling to provide more space and a more reclined seated position, which is suitable for larger and heavier and/or pain-sensitive users. BariBar is very easy to use. The built-in adjustment function automatically adapts the sling bar to each user for optimal comfort. Reliable safety latches prevent the lifting sling straps from creeping out of the sling bar.

## Check product function

---

### Visual inspection

Check product function regularly. Ensure that the material is free from damage. If damage is apparent, the product must be discarded.

## Always read the manual

---

Always read the manuals for all assistive devices used during the transfer.

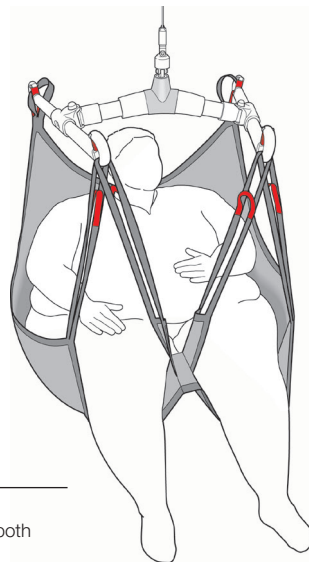
Keep the manual where it is easily accessible for users of the product.

Never leave a user unattended during a transfer.

## Product description BariBar

---

BariBar is made of aluminium and steel. The lift hooks are equipped with safety latches. BariBar must be used in combination with ceiling lift units that are recommended by Handicare.

All four hooks must always be used.

## Operation

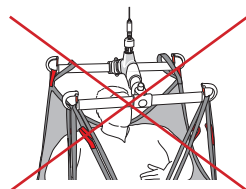
---

When moving the BariBar without any load applied, hold the side bar with one or both hands to prevent it from swinging.

When load is applied, use both the sling bar and the lifting sling to manoeuvre.

### Lifting with BariBar

1. Use an approved lifting sling.
2. Place the BariBar with the main bar directly over the user and with the hooks marked with head symbols pointing towards the users back and head.
3. First, connect the lifting sling's back loops.
4. Then, connect the leg support loops.
5. Apply tension and ensure that everything is correct before starting to lift. If the user is uncomfortable, stop lifting.

## SWL

---

The maximum load for BariBar is 454 kg.

Different products in the complete lift system (lift unit, sling bar, scales and other accessories) may be rated for different max. loads. The lowest rated max. load always determines the max. load for the entire system. Always check the max. load for the lift unit and lifting accessories before use, and contact your dealer when in doubt.

## Cleaning

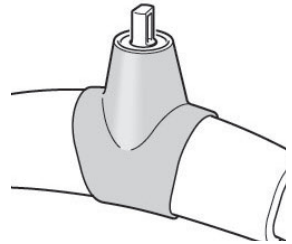
---

If necessary, clean with warm water or a soap solution. Do not use cleaning agents containing phenol or chlorine, as this could damage the aluminium and the plastic materials. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

## BariBarWrap

---

BariBar must always be fitted with BariBarWrap, a wipeable protection cover for secure handling of the sling bar.


## Installing BariBar

Lower the lift strap and loosen the two screws on either end of the threaded pin. Insert the BariBar's lift point into the H-adapter and replace the threaded pin. Replace the two screws with the lock washers. Use two 4 mm Allen keys to tighten the screws.


Installation of BariBar on an overhead lift unit

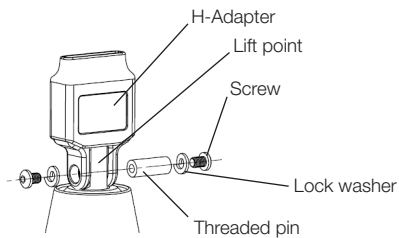- 1 x 90002955 (Threaded pin)
- 2 x 90001656 (Lock washer 6.1x11.8x1.6)
- 2 x 90001621 (Screw K6S M6x8)
- 2 x Allen key 4mm


RiseAtlas/RiseBasic

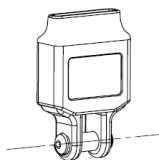
1


2

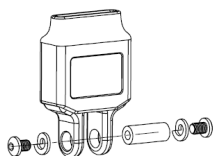3

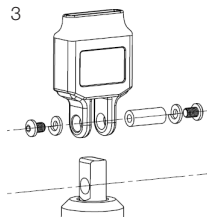4
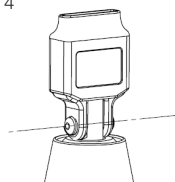
## Technical information

---

**Weight:** 8.6 kg /18.9 lbs

**Exterior dimensions:** 197 x 682 x 609 mm, 7.7 x 26.8 x 23.9 inches

**Lift dimensions, c-c lift loops:** 600 x 546 mm, 23.6 x 21.4 inches

**Colour:** RAL 9006

**Material:** Steel, aluminium, plastic

**SWL:** 454 kg, 1000 lbs

**Angular displacement, side bars:** 45°

**Min. distance lift hooks, c-c lift loops:** 366 mm / 14.4 inches

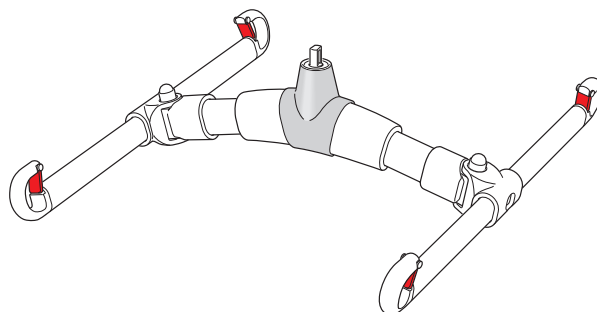
**Max. distance lift hooks, c-c lift loops:** 720 mm / 28.3 inches

## Compatibility

---

	Reda	RiseBasic	RiseAtlas
HighbackSling M-XXL			
ShellHBSling M-XXL			
FlexibleSling (Divided leg supports) M-XL			

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 = combinaciones recomendadas

## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

**handicare**

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)