# Protective Cover Disposable Cover

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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Protective Cover

Disposable Cover

REF
-----

1051	1055
1151	1155
1251	1255
1451	2055
1481	9138
2051	9168
2052	

**SWL for protective cover for EasySlide: 300 kg/660 lbs**

**SWL for protective cover for RollerSlide: See board**

## Table of contents

<b>English</b> .....	<b>3</b>
<b>Svenska</b> .....	<b>4</b>
<b>Norsk</b> .....	<b>5</b>
<b>Dansk</b> .....	<b>6</b>
<b>Suomi</b> .....	<b>7</b>
<b>Deutsch</b> .....	<b>8</b>
<b>Nederlands</b> .....	<b>9</b>
<b>Français</b> .....	<b>10</b>
<b>Italiano</b> .....	<b>11</b>
<b>Español</b> .....	<b>12</b>
<b>Product information</b> .....	<b>13</b>

# Protective Cover Disposable Cover

SystemRoMedic™

English

Different protective covers are available for SystemRoMedic's EasySlide and RollerSlide. Protective Cover is a multi-use cover that is made of Ultra Low Friction (ULF) material. Disposable Cover, a single-use cover that is made of plastic, can be used to advantage in e.g., emergency care or on isolation wards. The multi-use protective cover for RollerSlide is also a glide cover, since it enables the slide to glide freely.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

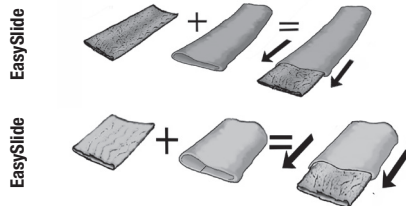
## Always read the user manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product. Always make sure that you have the right version of the manual.


The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

## Placement

Disposable Covers are on convenient rolls and are torn off one by one.

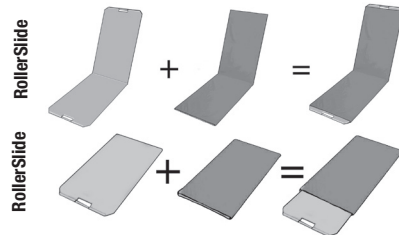Disposable Cover and Protective Cover are pulled over EasySlide or RollerSlide (see images).

When using Disposable Cover on RollerSlide, remember that a glide cover is always needed, since it is what allows the slide to glide freely.

**NOTE!** DisposableCover cannot be laundered, but must be discarded after use.

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

## Expected lifetime

Up to 10 years of normal use

# Protective Cover Disposable Cover

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

**handicare**

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)