IncoSheet

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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REF

1530

1531

1532

1533

SWL: 300 kg/660 lbs

Table of contents

Size	3
English	4-5
Svenska	6-7
Norsk	8-9
Dansk	10-11
Suomi	12-13
Deutsch	14-15
Netherlands	16-17
Français	18-19
Italiano	20-21
Español	22-23

Size

Art no.	Productname	Size mm/inch	Description
1530	IncoSheet	750 x 900/ 29.5 x 35.4	
1531	IncoSheet	750 x 2000/ 29.5 x 78.7	With fabric on both sides
1532	IncoSheet	750 x 1500/ 29.5 x 59	
1533	IncoSheet	400 x 830/ 15.7 x 32.6	

IncoSheet is a washable incontinence barrier that can be used in bed or on a wheelchair seat cushion. The material is highly absorbent and distributes fluid over a large area. The fabric on the top side of the product makes the surface feel dry and comfortable. The rough fabric on the underside provides friction to keep the sheet in place in the bed.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

Always read the manual

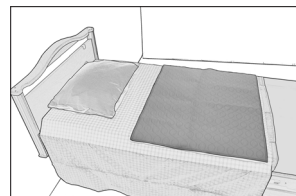
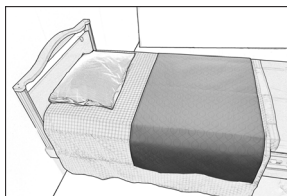
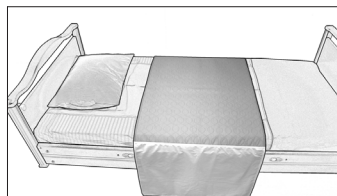
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.


The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Placement


Place the IncoSheet with the quilted, lighter side facing upwards in the bed.

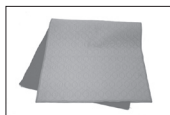IncoSheet with the longer incontinence barrier is mainly intended for users who need help with turning. This size model can be placed on top of the transfer assistive device, since it has an extra long incontinence barrier and continues to provide protection, even after turning.

750 x 1500 mm/
29.5 x 59"

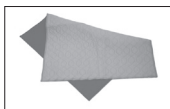The two shorter incontinence barriers are mainly intended for users who are able to turn themselves, but they can also be used when the user requires assistance to turn, in which case a transfer assistive device is placed on top. Since the transfer assistive device is placed on top of the IncoSheet, the IncoSheet remains in the same place under the user during and after turning, thus continuing to provide incontinence protection.


750 x 900 mm/
29.5 x 35.4"

750 x 2000 mm/
29.5 x 78.7"

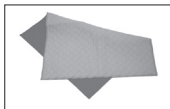The small model, IncoSheet 40 x 830, can be placed in a wheelchair to protect the seat cushion.

400 x 830 mm/
15.7 x 32.6"

Material

Polyester, cotton

Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

handicare

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com