PositioningCover

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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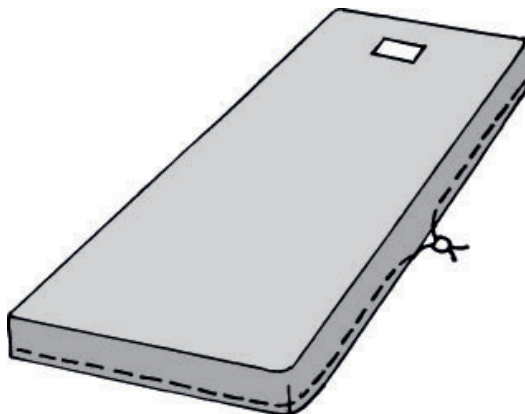
REF

1596

SWL: 300 kg/660 lbs

Table of contents

English	4-5
Svenska	6-7
Norsk	8-9
Dansk	10-11
Suomi	12-13
Deutsch	14-15
Nederlands	16-17
Français	18-19
Italiano	20-21
Español	22-23

PositioningCover is a mattress cover with drawstring that is used under a regular sheet for positioning in bed. PositioningCover can be used with all types of mattresses and in situations where there are special requirements for user mobility in bed, for example, for bedridden users in intensivecare units.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the user manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

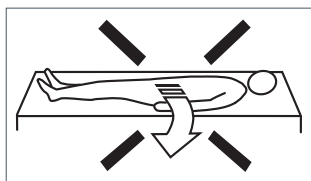
Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.handicare.com.


Important information

Never leave the user unattended, since the mattress cover is very slippery. Use bed rails, since there is a risk of the user sliding out of bed. Do not allow the user to sit up in bed unassisted.


Placement

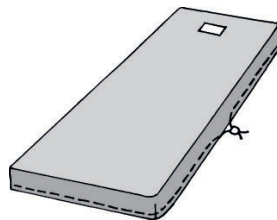Lay the mattress cover on the mattress with the slippery side facing up, as indicated on the product label. Place an extrawide sheet (or two sheets crossways) over the mattress cover.


Turning

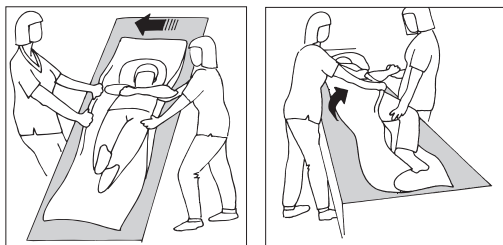Move the user to one side of the bed before turning. Take care not to move the user too close/over the edge of the mattress. Turn the user on one side with the help of the sheet. Prevent involuntary sliding by tucking the sheet securely under the mattress.

Higher up

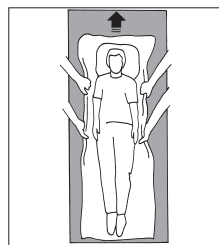Position the user higher up by pulling on the sheet (see image). If the user is heavy, a slide sheet can also be placed under the user.

Material

Polyamide

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumble-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

handicare

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com