Instructions for use - English

Bruksanvisning - Svenska

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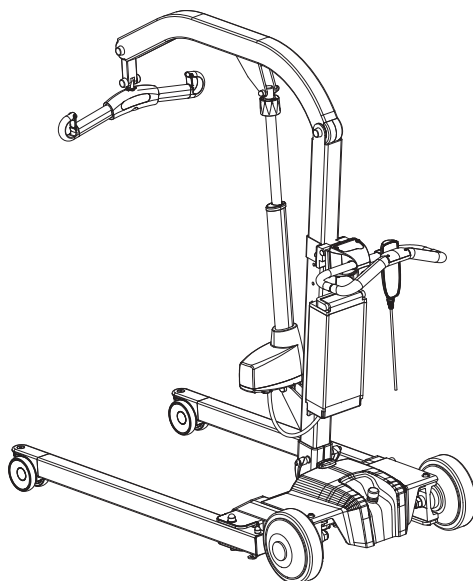
REF

60900002

60900003

**SWL: 205 kg/450 lbs**  
**270 kg/600 lbs**


## Table of contents
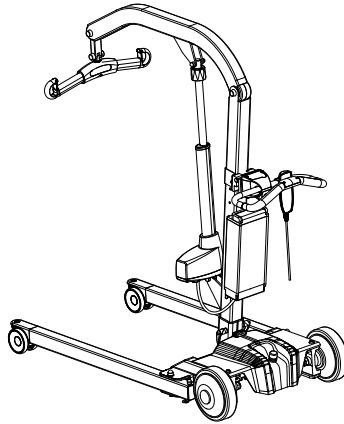
<b>English</b> .....	4
<b>Svenska</b> .....	18
<b>Norsk</b> .....	32
<b>Dansk</b> .....	46
<b>Suomi</b> .....	60
<b>Deutsch</b> .....	74
<b>Nederlands</b> .....	88
<b>Français</b> .....	102
<b>Italiano</b> .....	116
<b>Español</b> .....	130
<b>Dimensions</b> .....	143

# EvaDrive

---

EvaDrive a motor of Eva mobile lifts and is available as an accessory. It has been developed to meet the increasing need for power assistance when maneuvering a user in the mobile lift. EvaDrive unit supplies the power by its motorised wheels and is controlled by the unique intuitive handle bar.

## Safety information

---

### Visual inspection

Inspect lift functions regularly. Check to ensure that material is free from damage.

### Before use

Make certain the lift is properly assembled.

Check lifting function and base-width adjustment.

Check driving function in all directions.

Check that the accessory (EvaDrive) is properly mounted on the lift.

Check sling bar connection and safety latch function.

Check to ensure that the quick-connecting locking pin for the actuator is correctly installed.

## Always read the user manual

---

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

Under no circumstances may a lift with EvaDrive be used by persons who have not received instruction in the operation of a lift with the accessory mounted.

It is strongly prohibited to modify the original product.

## Intended use of the product

EvaDrive in an add-on system to Handicare Eva Mobile lifts in order to provide a power assisted drive function to the rear wheels. The EvaDrive shall be used in the following situations and environments:

- In a passive patient lift situation with a user in the lift, reducing the effort from the caregiver when maneuvering the lift.
- In a dry environment and on a level surface.
- Home Care environments.
- Healthcare environments.

## Assembly

### Unpacking and inspection

#### Check to ensure that all components are included

Mast with lift arm, lift motor, sling bar.

EvaDrive undercarriage with base-width adjustment motor and locking handle.

Hand control and cord.

Handlebar

Allen key

2 screws + bracket for handlebar

Instructions, battery, charger and charger cable.

Check to ensure that no components are left in the packaging.

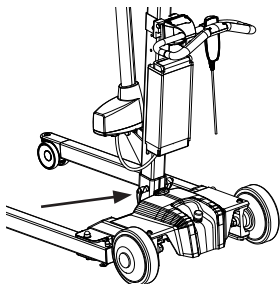
Inspect the drive unit for signs of damage.

Periodic inspection must be performed at least once yearly. Use only original spare parts. Service and maintenance must be performed according to instructions in the manual.

The drive unit must be charged for at least 10 hours before it is used for the first time.

Install the drive unit on the lift according to the accompanying assembly instructions. (Separate document) Assembly must be performed by Handicare authorized personnel.


#### Installation of mast and battery

Insert the mast onto the base. There are three alternatives for adjusting the height of the mast. Secure the mast with the locking handle.

Slide the battery into the rails on the backplate mounted on the mast.

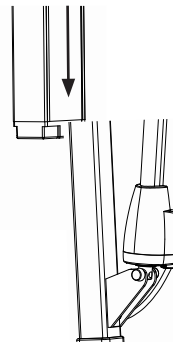
Handle the battery with care. Do not drop.

### Installation of the handlebar




The handlebar must be installed on the mast at a height of at least 33.4 inch from the floor. Measure from the lowest point of the handlebar.

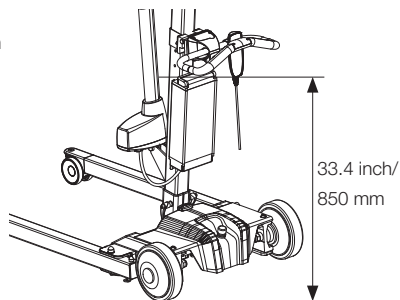The handlebar is secured using the accompanying Allen key.

### Installation of cables

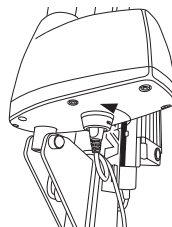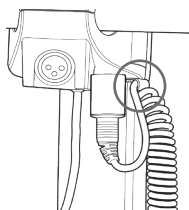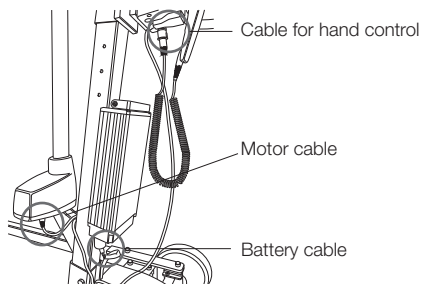Connect the battery cable and the cable for the hand control. Secure the hand control cable with the retainer. Connect the motor cable and turn to lock in place.

## Final inspection

- Check to ensure that no components are left in the packaging.
- Inspect the product for signs of damage.
- Inspect wheels and brakes.
- Check mechanical brake function by activating the brakes and, without activating EvaDrive, try to move the lift forward. If the lift does not move, the brakes function as they should.
- Check all connections, screws and bolts.
- Check emergency-stop function by activating EvaDrive; press the emergency-stop button, and then press the up or down button. If nothing happens, the emergency stop is functioning as it should, since it stops all electrical functions.
- Using the hand control, activate EvaDrive, press the up button and run the lift boom all the way up. Then, press the down button and run the boom all the way down.
- Test base-width adjustment. Activate the system, press the button for base-width adjustment, open the undercarriage to maximum width, and then press the other button to reduce the width.
- If the lift is functioning as it should, connect the charger and check to ensure that the red charging indicator light on the charger lights up.
- Activate EvaDrive and grasp the handlebar with both hands, hold the safety switch in, and then run the lift forward, backward and in a turning motion. Check that the lift moves easily, to both the right and to the left.
- Test the rear wheels by turning them a bit; turn the lift back and forth. Turn the wheels all the way and run the lift sideways. Check that the yellow light lights up when the wheels turn and are turned, and that the light goes out when the wheels are returned to the normal position.
- Test lift function by lifting a person (not a user) using an approved sling. At the same time, check the emergency lowering function with someone on the lift. See section on Emergency lowering.

### NOTE!

The lift must be charged for at least 10 hours before it is used for the first time. See section on battery charging. Keep the manual where it is easily accessible for users of the product.

# Using the product

## Important information

---

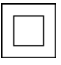
- The lift must be assembled according to the assembly instructions included with the product.
- The lift may only be operated indoors on a dry and level surface.
- Never leave a user unattended during a transfer.
- Maximum load may under no circumstances be exceeded. See section on maximum load.
- The lift must not be lowered into water or used in the shower.
- The lift must not be left or stored in a damp or humid environment.
- The lift must not be charged in a wet room.
- For optimal function, inspect the lift regularly. See section on Maintenance.
- Warranty applies only if repairs or alterations are made by personnel who are authorized by Handicare.
- Never attempt to lift the mobile lift unit by the handlebar.
- Ensure that there are no obstacles or people in the way of the lift.
- Never stand on the lift during operation. Always walk behind the lift.
- Low speed is recommended when a user is seated in a sling attached to the lift.
- The lift must not be used in oxygen-enriched environments.
- Handle the battery with care. Do not drop.
- Use only batteries, cables and chargers intended for EvaDrive.
- If a lift is attempted at an angle, a safety function will temporarily pause the lifting motion. Ensure that the EvaDrive is correctly positioned over the user, or let the EvaDrive position itself with the brakes unlocked. Once the lift is in its correct position, the lifting motion will automatically be resumed.
- Lifting accessories must be properly fitted and tested in relation to the user's needs and functional ability.
- Never move the lift by pulling on the actuator!
- Using lifting accessories other than those approved can entail a risk.
- Verify that all lifting accessories hang vertically and can move freely.
- Activate the brakes whenever the lift is not in use.

## Safe working load

Different products on the same lift system (lift unit, sling and other lifting accessories) may have different allowable safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your local Handicare representative or dealer if you have any questions.

## Description of functions and symbols

EvaDrive has many built-in functions. Each function is described in the respective section in the user manual, or in the user manual for the respective accessory. Below is a brief description of the functions that are available for caregivers and users.

	Blue/white Read user manual		May not be discarded in domestic waste
	Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.		Type B, according to the degree of protection against electric shock.
	The device is intended for indoor use.		Class II equipment
	Tested according to: ANSI/AAMI HA60601-1-11:2011, ANSI/AAMI ES60601-1:2005/(R)2012, CAN/CSA-C22.2 no. 60601-1:14		Yellow/Black International General Warning Symbol
	Yellow/Black Warning. Pinch point		

## Hand control

### Raising/lowering the lift arm

Symbol indicate direction of travel.

Motion stops as soon as the button is released.

If the lift arm encounter an obstacle while lowering, the lift will immediately stop the motion.

In order to continue, the lift arm will need to be raised slightly by using the hand control before further lowering.

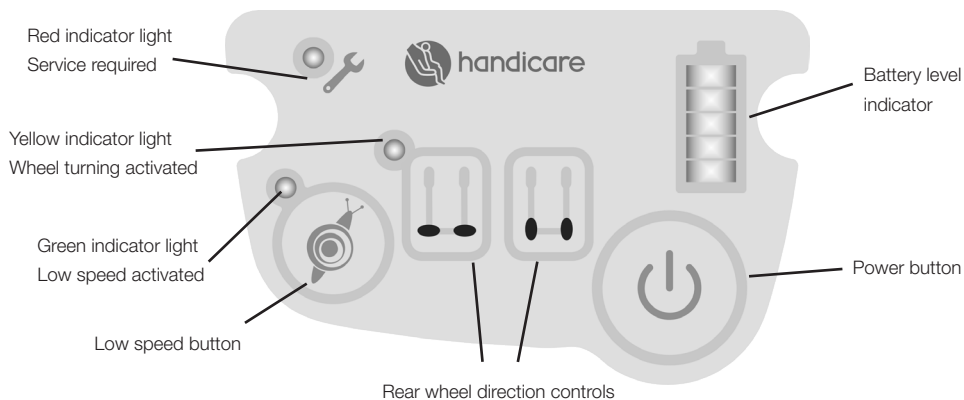
### Electrical base-width adjustment

Markings on the buttons of the hand control indicate function.

Motion stops as soon as the buttons are released.

## Information panel

**Red indicator light** - Solid red light indicates that service is required. Blinking at the same time as one of the lights on the battery scale: error code, see troubleshooting. Blinking at the same time as the lights on the battery scale are not on indicates that the battery must be charged.

**Yellow indicator light** - Solid light indicates rear wheels are turned. The lift can then only be moved sideways. Light turned off indicates that the rear wheels are in normal position and the lift can be driven forward/backward.

**Green indicator light** - Solid light indicates low speed. Blinking indicates the need for calibration. (Switch off, wait for 10 seconds and switch back on.) No light, normal condition.

**Battery level** - Solid light indicates battery capacity remaining. Blinking light indicates error. A service technician can determine the source of the error.

**Power button** - Press lightly to activate EvaDrive. (Wait about 2 seconds before operating.) Press again to deactivate EvaDrive.

**Low speed button** - Press once to set the system for low speed (green light switches on); press again to resume normal operation (light switches off).

**Rear wheel direction controls** - Press the left button to set the rear wheels at an optional angle. Press the right button to resume normal wheel setting. Motion ceases as soon as pressure on the buttons is released.


## Preventative service/maintenance

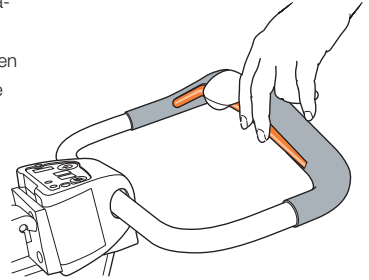The symbol for service/maintenance lights up automatically when the lift unit has been in service for 12 months. Contact an authorized Handicare service technician for maintenance and resetting.

## Operation

Press the power button once to activate EvaDrive. The lift is ready for operation after about 2 seconds.

No load should be applied to the handlebar or the orange safety switch when the lift is being started; calibration takes place instantaneously when the system is starting up. Grasp the handlebar, including the safety switch, and apply pressure to the handlebar in the desired direction of travel; the lift will then begin to move. The handlebar senses the desired speed and direction of travel, and adapts to the right level. Transfer/movement of the lift is more gentle and more even if the arms are somewhat relaxed while manoeuvring the lift. To turn, simply apply a light pressure to the handlebar in the desired direction.


**NOTE!** Only minimal pressure is required to manoeuvre EvaDrive, regardless of the load on the lift.

## Using the parking brakes

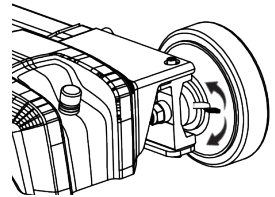The parking brakes are activated by being pushed down and deactivated by being pushed up wards.

When the brakes are activated they are automatic and only switched on when the lifts is standing still and are released when a signal is given trough the handlebar. A click is heard when the brakes are switched on or off. When the brakes are deactivated the lift can be moved manually even though it moves easier if the drive system is shut off.

It is recommended that the brakes are deactivated during lifts.

## Safety check before starting

After starting the lift, do not touch the handlebar for a few seconds to allow the lift to automatically calibrate. While this is happening, the green light beside the low-speed button will blink quickly; operation of the lift is not possible until EvaDrive has completed automatic calibration.

If the handlebar is hit violently or subjected to impact, the sensors' neutral values can be affected so much that the blinking warning will not stop. In this case, manual calibration must be done according to the following instructions.

## Manual calibration of EvaDrive

Hold the orange safety switch in while EvaDrive is starting up. This will cause the green light to blink slowly. Continue to hold the safety switch in for about 5 seconds until the green light starts to blink faster. Release the safety switch and do not touch the handlebar until the green light stops blinking (after about 10 seconds).

## Troubleshooting

The green light blinks, but stops after a while.	Automatic calibration complete
The green light blinks.	<ul style="list-style-type: none"> <li>- Manual calibration is required, see section on manual calibration.</li> <li>- Check to ensure that no load was applied to the handlebar when the lift was started e.g., that nothing was hanging on the handlebar. In that case, remove it and the green light will stop blinking.</li> </ul>
All or parts of the battery scale blink quickly.	Something is wrong with the lift's electrical system. Restart EvaDrive. If there is still a problem, contact Handicare.
Red light + battery scale blink	<ul style="list-style-type: none"> <li>- Restart</li> <li>- Contact Handicare</li> </ul>
If unusual sounds are heard:	<ul style="list-style-type: none"> <li>- Try to locate the source of the sound.</li> <li>- Take the lift unit out of service and contact Handicare.</li> </ul>
The lift cannot be started	<ul style="list-style-type: none"> <li>- Check the battery switch.</li> <li>- Check battery connection.</li> <li>- Connect charger and charge for at least 2 hours.</li> <li>- Contact Handicare.</li> </ul>
If the lift or base-width adjustment cannot be activated, check the following:	<ul style="list-style-type: none"> <li>- That the emergency stop button is not pressed in.</li> <li>- That all cables are properly and securely connected.</li> </ul> <p>Pull out the contact and plug it in again firmly.</p> <ul style="list-style-type: none"> <li>- That battery charging is not in progress.</li> <li>- That the battery is charged.</li> </ul> <p>If the lift is not working properly, contact your dealer.</p>

## Optimal pattern of operation

### Normal travel in corridors and open areas

Ensure that the rear wheels are in the normal position, see Fig. 1. Grasp the handlebar and hold in the safety switch, then move the lift. The lift will sense both the desired direction of travel and speed. The lift has a greater turning radius at this setting.

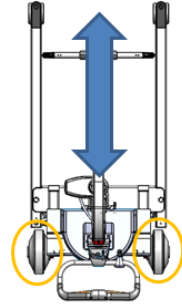
Figure 1. Wheels adjusted for forward motion

**When manoeuvring in confined areas** the rear castor wheel function can be used to advantage. Manoeuvre the lift as far as possible at this setting, then press the button for rear wheel adjustment and hold it pressed in until the desired wheel angle is obtained, see Fig. 2. The yellow LED light on the control panel indicates that the wheels are in the turned position. Grasp the handlebar and hold in the safety switch, then turn the lift in the desired direction to rotate.

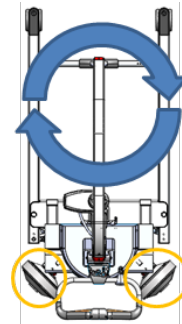
Figure 2. Rotation

If the button for rear wheel adjustment is held in until the wheels stop in their outermost position, the lift can be moved all the way to one side, see Fig. 3; This setting can be very useful for repositioning the user further up in bed, for example. If the low-speed symbol is pressed, the max. speed is limited, thereby enabling greater precision in confined spaces.

**NOTE!** Be aware of the pinch point when the rear wheels are turning.

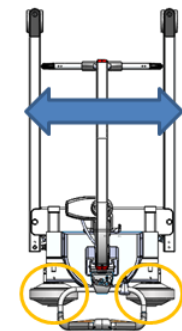
Figure 3. Lateral travel

# Battery charging

Always charge the lift after use to ensure that the battery is fully charged.

Charging procedure:

- Connect the charger cable to the batterypack, plug in the wall socket and switch on at plug socket.
- Check to ensure that the light on the charger lights up and that the battery's power switch is switched on!
- Charging stops automatically when battery is fully charged.

1. The green LED light confirms that the charger is connected to the power outlet and is receiving power.
2. Red light indicates charging.
3. Green light indicates full charge.

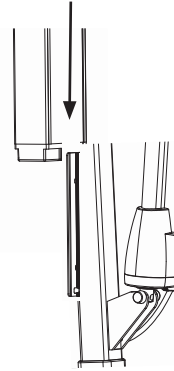
**NOTE!** The lift must be charged for at least 10 hours before it is used for the first time. For maximum service life, charge batteries regularly. We recommend daily charging when the lift is used daily. Only the dedicated charger may be used.

# Changing the battery

Loosen the two screws that holds the handlebar using an Allen key, and move up wards as far as possible. Remove the battery cable. Slide the battery off the rails on the on the backplate mounted on the mast.

Slide the new battery into the rails. reattach the handlebar and the battery ca


Handle the battery with care. Do not drop.

# Emergency stop STOP

**Emergency stop:** Press the emergency-stop button on the EvaDrive cover. Emergency stop switches off all of the lift's electrical functions. The battery scale continues to light continuously.

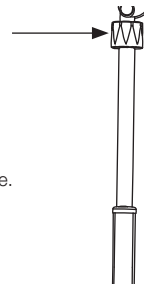**Resetting:** Turn the emergency-stop button in the direction of the arrows until the button pops out.

# Emergency lowering

## Manual emergency lowering

For manual emergency lowering, turn the round plastic knob on the actuator clockwise.

## Accessories

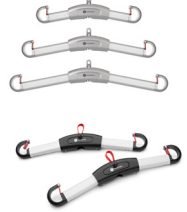
### Lifting slings

Handicare's SystemRoMedic™ line includes a wide range of functional and comfortable, high-quality lifting slings that are adapted for all types of lifting and for users with different needs. The lifting slings are available in several materials and in sizes ranging from XXS to XXL. There are also special lifting sling models in XXXL and XXXXL for extremely large and heavy users. All models are safe and very easy to use.

### Sling bars

SlingBar is a two-point aluminium sling bar available in four variants with different width. All variants of SlingBar have safety latches which prevent the sling straps from creeping out of the sling bar and all variants are for users weighing up to 300 kg/660 lbs. To get more room in a lifting sling when using SlingBar two-point sling bar, SlingBarSpreader M side bars can be used as an accessory. SlingBarSpreader M open up the lifting sling and provide for a more reclined position.

SlingBar XS article no.: 70200071

SlingBar S article no.: 70200001

SlingBar M article no.: 70200002

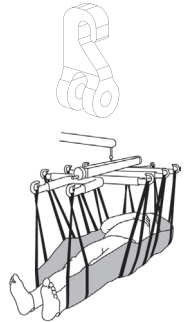
SlingBar L article no.: 70200003

SlingBarSpreader M article no.: 70200042

StretcherBar, article no.: 70200006, and StretcherSling, article no.: 46502007, for lifting in a supine position.

Sling bar RFL X4, Article no.: 70200017, is a four-point sling bar designed to provide more space in the sling, for example, for obese and/or pain-sensitive users.

SwiftHook for sling bar article no.: 70200008

### Hand control

Hand control HB33-6, article no.: 70200089, with service, battery status and overload indicator.

### Scales

Charder MHS2500 is used together with a stationary or mobile lift for weighing of users.

Article no.: 70100002, SWL: 300kg/660lbs

Article no.: 70200003, SWL: 400kg/880lbs

### Assistive devices for positioning

Handicare's SystemRoMedic™ range includes a wide selection of functional, comfortable, high-quality assistive devices for positioning that can be adapted for different types of lifting and for users with different needs.

# Maintenance

The lift must undergo thorough inspection at least once per year. Inspection must be performed by authorized personnel and in accordance with Handicare's service manual.

Repairs and maintenance may only be done by authorized personnel using original spare parts.

Used batteries are to be left at the nearest recycling station. Used batteries can also be returned to Handicare or a Handicare dealer for recycling.

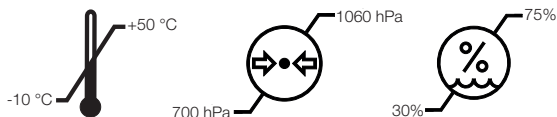
## Cleaning/disinfection

If necessary, clean the lift with warm water or a soap solution and check that the castors are free from dirt and hair. Do not use cleaning agents containing phenol or chlorine, as this could damage the aluminium and the plastic materials.

If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used. Disinfect the parts that come in contact with a user before the lift is used for the next user.

## Storage and transportation


If the lift is not to be used for some time or e.g., during transport, we recommend that the emergency stop button be pressed in. The lift should be transported and stored in -10 ° C to + 50 ° C and in normal humidity, 30% -75 %. The air pressure should be between 700 and 1060 hPa. Let the lift reach room temperature before the batteries are charged or the lift is used.

## Service agreements

Handicare offers service agreements for maintenance and regular testing of your lift unit. Contact your local Handicare representative.
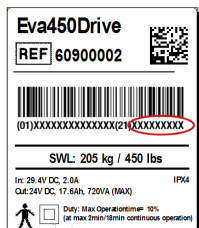
## Serial number

## Technical information

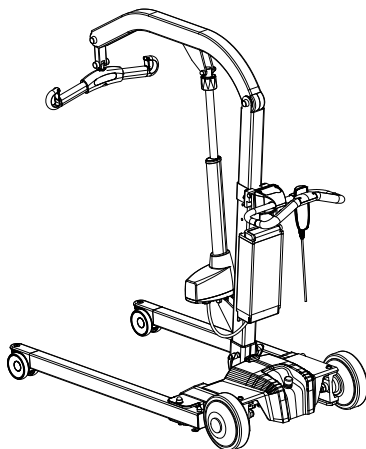
<b>Lifting speed</b>	27,2 mm/s without load.
<b>Maximum speed (normal)</b>	Forward: 4.3 mph / 7 km/h, Backwards: 2.1 mph / 3,5 km/h
<b>Maximum speed (low speed )</b>	2.1 mph / 3,5 km/h
<b>Batteries</b>	24V, 17,6 Ah Li-Ion
<b>Charger</b>	Modiary MDA10129402000. Output: DC 29, 40 V, 2.0 A
<b>AC power input cable</b>	HD3VV-F 3G0,75mm <sup>2</sup> . TCA090812EA. Length 59 inch, 1500 mm
<b>Motor (mast)</b>	DC 24 V, 10 A. IP X4. Operationtime: 10% at maximum continous operation of 2 minutes, maximum 5.5 cycles per minute. Push: 12 000N.
<b>Motor (base)</b>	24 V, 3 A, IP X4. Operationtime: 10% at maximum continous operation of 2 minutes, maximum 5.5 cycles per minute. Push: 3000N.
<b>Motor (EvaDrive)</b>	24 V, 16 A, IP X0. Operationtime: 10% at maximum continous operation of 2 minutes, maximum 5.5 cycles per minute. Push: 900N.
<b>Motor cover</b>	Flame-resistant ABS plastic
<b>Sound level</b>	With load: upwards: 74.7 dB(A) downwards: 52.6 dB(A).
<b>Intermittent operation</b>	Op 10/90, active op. max. 2 min. Out of a time of 100, active time must be less than 10, though not more than 2 min.
<b>Nav motor</b>	24V DC Mono Shaft, 180 W
<b>Material</b>	Aluminium/steel
<b>Emergency lowering</b>	Manual
<b>Castors</b>	Front: 4 inch, 100 mm, rear: 8 inch, 203 mm
<b>IP class</b>	IP X4*
<b>Weight</b>	Eva450Drive: 127.87 lbs, 58 kg Eva600Drive: 141.1 lbs, 64 kg
<b>Weight (heaviest part)</b>	Eva450Drive: 97 lbs, 44 kg Eva600Drive: 110.2 lbs, 50 kg
<b>Max. load</b>	EvaDrive450: 450 lbs, 205 kg, EvaDrive600: 600 lbs, 270 kg
<b>Turning radius</b>	Eva450Drive: 47,5 inch/ 1207 mm Eva600Drive: 57,5 inch/ 1307 mm
<b>Expected service life</b>	10 years
<b>Pressure control panel</b>	4 N
<b>Clearance</b>	1.1 inch, 28 mm

\*IP X4: Splashing of water, Water splashing against the enclosure from any direction shall have no harmful effect.  
(Test duration: 5 minutes Water volume: 10 litres per minute Pressure: 50–150 kPa)

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

## EMC

This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as reorienting or relocating the equipment/system or shielding the location.

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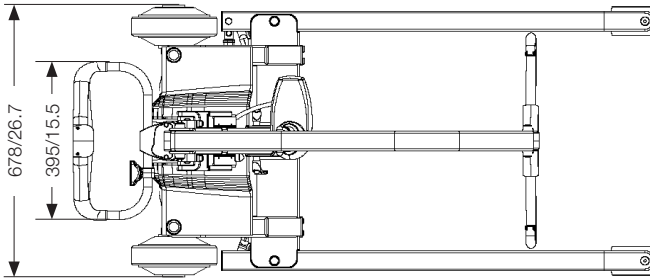
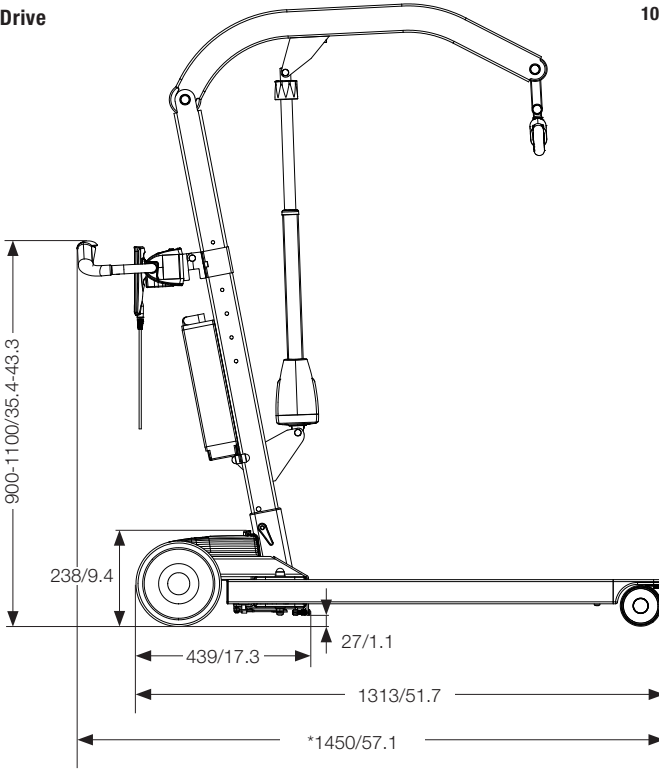
# Dimensions

All measurement are in mm/inch  
Tolerance +/- 5 mm/ 0.2"

## Eva450Drive



\*When the handle is placed  
1000 mm from floor

All measurement are in mm/inch  
Tolerance +/- 5 mm/ 0.2"

\*When the handle is placed  
1000 mm from floor

### Eva600Drive

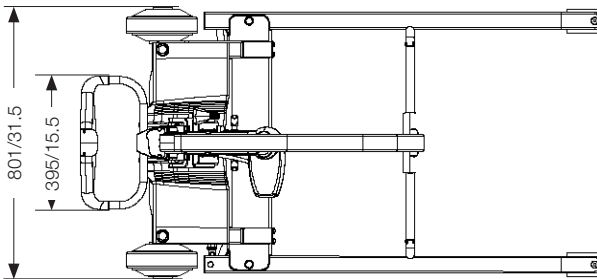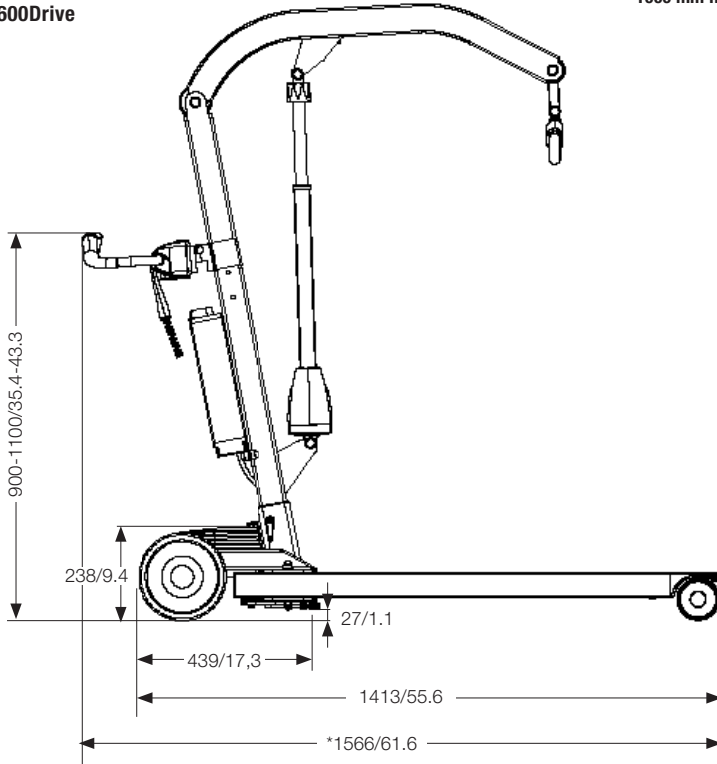**Guidance and manufacturer's declaration – electromagnetic emissions**

The EvaDrive is intended for use in the electromagnetic environment specified below. The customer or the user of the EvaDrive should assure that it is used in such an environment.

<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
<b>RF emissions CISPR 11</b>	<b>Group 1</b>	<b>The EvaDrive uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.</b>
RF emissions CISPR 11	Class A	The EvaDrive is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

### Guidance and manufacturer's declaration – electromagnetic immunity

The EvaDrive is intended for use in the electromagnetic environment specified below. The customer or the user of the EvaDrive should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6 kV contact +/- 8 kV air	+/- 6 kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient / Burst IEC 61000-4-4	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/- 1 kV differential mode +/- 2 kV common mode	+/- 1 kV differential mode +/- 2 kV for common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % $U_T$ (>95 % dip in $U_T$ ) for 0,5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  <5 % $U_T$ (>95 % dip in $U_T$ ) for 5 sec	<5 % $U_T$ (>95 % dip in $U_T$ ) for 0,5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  <5 % $U_T$ (>95 % dip in $U_T$ ) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the [Equipment or System] requires continued operation during power mains interruptions, it is recommended that the [Equipment or System] be powered from an uninterruptible power supply or battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

**NOTE**  $U_T$  is the a.c. mains voltage prior to application of the test level.

## Guidance and manufacturer's declaration – electromagnetic immunity

The EvaDrive is intended for use in the electromagnetic environment specified below. The customer or the user of the EvaDrive should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the EvaDrive, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	<p><b>Recommended separation distance</b></p> $d = 1,2\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80MHz to 2,5GHz	3 V/m	$d = 1,2\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3\sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$
EN-ISO 10535:2006	10 V/m 800MHz to 2GHz	10 V/m	<p>where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p>
EN 61000-4-3: Extended the frequency range 2,5 GHz – 2,7 GHz. EN 61000-4-3: Extended test level of 10 V/m for frequency range 1,0 GHz – 2,0 GHz. EN 61000-4-8: Extended test level of 30 A/m.			<p>Interference may occur in the vicinity of equipment marked with the following symbol.</p> 

NOTE 1 At 80MHz and 800MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflected from structures, objects and people.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EvaDrive is used exceeds the applicable RF compliance level above, the EVADRIVE should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the EvaDrive.

<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

**Recommended separation distances between portable and mobile RF communications equipment and the EvaDrive**

The EVADRIVE is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the EvaDrive can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the EvaDrive as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2,3\sqrt{P}$
<b>0.01</b>	<b>0.12</b>	<b>0.12</b>	<b>0.24</b>
<b>0.1</b>	<b>0.38</b>	<b>0.38</b>	<b>0.73</b>
<b>1</b>	<b>1,2</b>	<b>1.2</b>	<b>2.3</b>
<b>10</b>	<b>3,8</b>	<b>3.8</b>	<b>7.3</b>
<b>100</b>	<b>12</b>	<b>12</b>	<b>23</b>

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**Note 1:** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**Note 2:** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

# Making everyday life easier

## More than 30 years in the service of simplicity

Handicare's unique easy transfer concept is the market's widest and most complete range of clever, easy-to-use and safe assistive devices for all types of patient transfers and manual handling requirements.

The philosophy behind the concept is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, we offer improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Designed and produced by Handicare

For more than 30 years, Handicare has offered simple solutions for great results, and an easier everyday life for both users and personnel in the care sector. During 1984-2010 under the name of RoMedic and, as of January 1, 2011 under the name of Handicare.

*Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.*

*The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide.*

*Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.*

Handicare AB  
Torshamnsgatan 35  
SE-164 40 Kista, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)