

SWL: 160 kg/352 lbs

BackBoardStraps have been designed for use in combination with the Ferno Scoop Stretcher 065 EXL to enable mechanical lifting with a manual stretcher in emergency care. The straps are mounted on the Ferno Scoop Stretcher according to these instructions. BackBoardStraps consist of four independent lift straps. There are 6 alternative lift loops on one side of each strap to facilitate balancing of the stretcher. The BackBoardStraps are looped securely through the handle holes (two on each side), and then connected to the slingbar hooks.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.


To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Application and use



The straps are secured through the holes on the sides of the stretcher at the position of the patient's hips and shoulders. Simply pull a strap through each hole, and then loop it through itself.

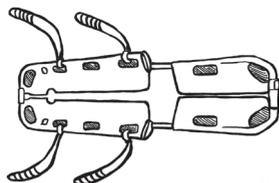

Two straps are mounted on each side of the stretcher, and then hooked to the slingbar. Adjust the slingbar to the appropriate width to prevent discomfort to the patient.



Before lifting, balance the stretcher using the six alternative lift loops.

Start lifting carefully, and then stop to ensure that the stretcher is properly balanced in a horizontal position. If it is balanced, continue lifting; if not, lower the lift and adjust the balance using the lift loops.

In each lifting situation, always perform an individual test to ensure that the lift is properly balanced.



Always make certain that the stretcher is horizontal!

Always use the safety belts according to the instruction guide for the Ferno Scoop Stretcher!

Before lifting:

Read and follow the instructions for the Ferno Scoop Stretcher.

Plan the lift and decide how many caregivers will be needed to assist to ensure safety.


Never lift higher than necessary.

Do not leave the patient unattended during a lifting situation.

Dimensions

A: 110 cm

B: 8 cm

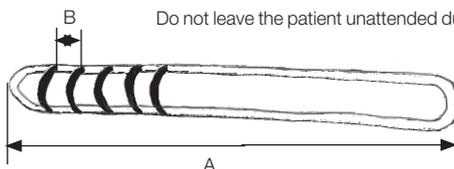

Care of the product:

Read the product label.



Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Handicare AB

Veddestav. 15. Box 640 Tel: +46 (0)8 -557 62 200

SE-175 27 Järfälla Fax: +46 (0)8-557 62 299

SWEDEN

Internet: www.handicare.com

E-mail: info@handicare.se

