Ready for life Manual - Shower seat

Item No. 12226
Max: 150 kg/330 lbs
Manual nr: 00747 Ver. 1 110921

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

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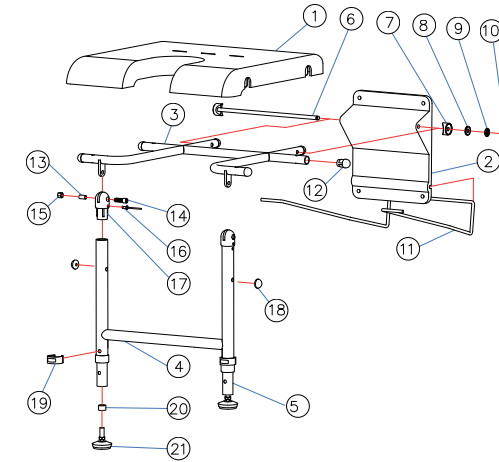
Manufactured for: Valentine International Ltd., Taiwan R.O.C.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

English

Svenska

Deutsch

Nederlands

Functional inspection

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use

Ensure that the product is correctly mounted/installed. Do a functional test by folding the shower seat up and down. Check that the seat surface is in a horizontal position when folded down.

Always read the manual

Always read the manual.
Keep the manual where it is accessible to users of the product.

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Installation instructions

- Seat height: a specialist must be consulted when setting seat height. Both feet of the user must reach the ground when seated. A normal chair can be used for reference. Measure the height of the seat surface and adjust to between 400-500 mm.
- Floorsupport: insert the legs on both sides at the correct height. Ensure that the catches locate firmly back into the hole. Check that the floor support is securly locked by pulling it.
- Mounting of the shower seat: mounting by a specialist is advised.
 - Select 6 stainless steel screws and washers and 6 wall plugs.
 - Determine the position of the seat (depth and height).
 - Mark the position of the 6 holes for the mounting plate.
 - Drill the holes and insert the wall plugs. Ensure horizontal alignment. The bracket must be in the recess between wall plate and wall.
 - Screw the wall plate tight.
 - Check the position of the floorsupport as described above in relation to the mounted shower seat. Both feet must be at the same length.

The mounting methods need to be determined at site and according to the type of wall. The distributor is not responsible for any incorrect mounting of products and the possible damage arising from incorrect installation.

Using the product

Fold the seat up and down by grasping the seat surface at the front. Move the seat down until the feet are on the floor. Aim for a seating position in the centre of the seat surface. Washing the intimate body parts without standing up is possible by using the U-opening at the front of the seat. After use, the seat can be folded up.

English	English
Deutsch	Deutsch
Nederlands	Nederlands
Svenska	Svenska

Maintenance

Clean regularly with normal household detergent. Do not use aggressive cleaners.

Technical data

Max. user weight:	150 kg/330 lbs
Seat depth	365 mm, 14.4"
Seat width:	410 mm, 16.1"
Seat height:	400-500 mm, 15.7-19.7" (in 5 steps of 25 mm, 0.1")
U-opening, width x depth:	170 mm x 140 mm, 6.7" x 5.5"
Material frame:	Aluminium and powder coated stainless steel
Material seat:	Plastic (PP)

Warning

- Risk of catching injury in the folding mechanism.
- Do not load the seat surface on one side only.
- Under no circumstance stand on the seat.

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