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EN - Please read the instructions before using the product.

These instructions apply to the soft backrests for Combi chairs.

Intended use

The soft backrests fits all Combi chairs with a maximum user weight of 150 kg.

The backrest is available in two variations:

- soft backrest standard size: part number 201130314
- soft backrest high: part number 201131314

The backrest is well suited for users who need a flexible and adjustable back support that can be adjusted to the individual. It is used in place of the standard backrest.

Installation

Remove the existing backrest by removing the clips on both sides (1). Lift up the backrest. Install the soft backrest by placing it down into the holes in the frame (2), securing it over the pins in the frame (3). Put both clips back into the holes in the frame (1).

Adjustment and use

The upholstery can be adjusted to the individual user by loosening or tightening the Velcro straps (4) on the back of the upholstery.

Be certain that the Velcro straps hold when the desired adjustment is achieved.

Maintenance

The upholstery on the backrest can be removed and machine washed at up to 80 degrees Celsius. It should not be dried in a tumble dryer. Remember to stick the Velcro straps together when washing.

The backrest can be cleaned with common cleaning solutions without abrasive or etching properties. It is not recommended to use solvents or chlorine.

Warning!

- Maximum weight of user: 150 kg.
- The backrest must only be used as described under the section "Intended use".
- Be certain that the Velcro straps hold after the upholstery has been adjusted.
- Be certain that the upholstery is balanced on the backrest, so that the Velcro can attach equally on both sides.
- Always be certain that the Velcro is sticking well and is tight.

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Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com