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## EN - Instructions for use

### Please read the instructions before you start using this product.

These instructions apply to the self propelled Combi chairs, in standard width as well as to the wide model. Combi chairs are equipped with a bedpan/ bucket holder.

### Overview and Use:

The Combi self-propelled is a very versatile chair, suited for semi-dependent or fully dependent individuals and those who require assistance with personal hygiene. It is also a good choice for wheelchair users with full use of their upper body, allowing for independent use in the bathroom and shower area.

Produced entirely of rust-proof materials and components, Combi includes 2 locking, plastic castors in the front and two 24 inch rubberized wheels with hand brakes in the rear. The chair can be used as a commode, a shower chair and an indoor transport chair for independent or caregiver assisted use. The Combi self-propelled chair features fold up armrests, useful during transfers. The polyurethane (PUR) seat, backrest, and armrests provide comfortable, anti-slip padding for the user.

### Assembly – see figure 1

#### Frame assembly:

Remove the snaplocks and mount the backrest into the holes in the frame. Replace the snaplocks, ensuring they are inserted correctly into the steel tube with both pins correctly placed in the holes. It is possible, although not necessary, to detach the backrest when the Combi chair is used over a toilet.

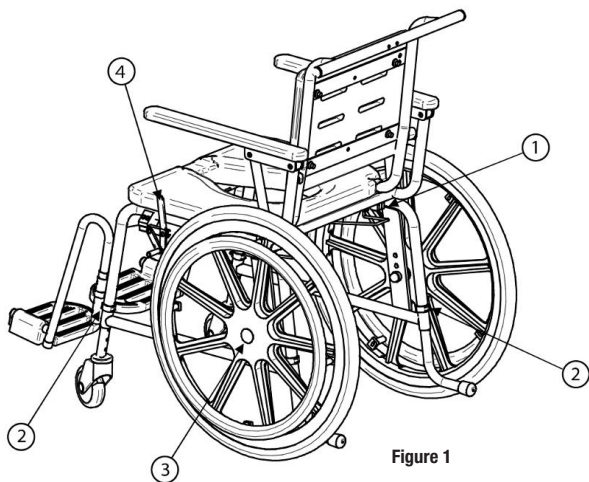
Figure 1

#### Assembling the front castors & rear wheels:

**FRONT castors:** Insert the inner-tube (castor) into the chair leg, aligning the larger hole of the castor tube with the hole in the leg tube. Insert the snaplock at the appropriate height with the black part facing to the front.

**REAR anti-tip feet:** Insert the inner-tube (anti-tip foot) into the chair leg, aligning the larger hole of the anti-tip tube with the hole in the leg tube. Insert the snaplock at the appropriate height (corresponding with the height of the front castor legs) with the black part facing to the back.

To ensure proper alignment of the castors, the snaplock pin will not fit into the tubes if the holes are mis-aligned.

**REAR wheels:** Remove the cover plate (3 on drawing) on the outside of the wheel. Unscrew the locking nut on the back of the wheel and remove the wheel axle. Choose the desired height position, replace the wheel axle, and re-insert and tighten the locking nut. Be certain that both the bolt and the nut are tightened. Replace the cover plate.

**REAR wheel brakes:** Loosen the screw (4 on the drawing) and move either forwards or backwards to suit the height of the wheels. The brake pad should be positioned approximately 1 cm/ 0.40 inches from the tire in the unlocked position. After adjustment, re-tighten the screw.

## Bedpan / Bucket frame – see figure 2:

The bedpan / bucket frame can be used in combination with a keyhole shaped or round bucket.

To assemble: The two end pieces (B, on the drawing) rest on the rear of the seat frame.

For keyhole shape bucket, insert the two front rods (A) into the inner holes (C). For all other bucket shapes use the outer holes (D). If no bucket or bedpan is being used, the frame may be detached so the chair fits easily over a toilet.

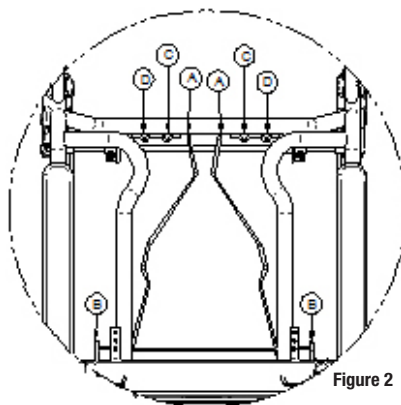
Figure 2

## Height Adjustment - see figure 3:

By repositioning the snaplocks on the chair legs, the Combi self-propelled may be adjusted to 6 different heights.

To adjust the height of the chair - see figure 3:

1. Remove snaplock
2. Select one of the 6 positions
3. Reinsert snaplock.

Repeat for all 4 legs (2 front castors and 2 rear anti-tip feet).

**IMPORTANT:** Check that all four legs are the same height and that all of the snaplocks are inserted correctly into the frame before use.

Adjusting the height of the rear wheels - see figure 1:

1. Remove the cover plate (3 on drawing) on the outside of the wheel.
  2. Unscrew the locking nut on the back of the wheel and remove the wheel axle.
  3. Choose the desired height position (3 choices)
  4. Replace the wheel axle, and re-insert and tighten the locking nut.
- Be certain that both the bolt and the nut are tightened.
5. Replace the cover plate.

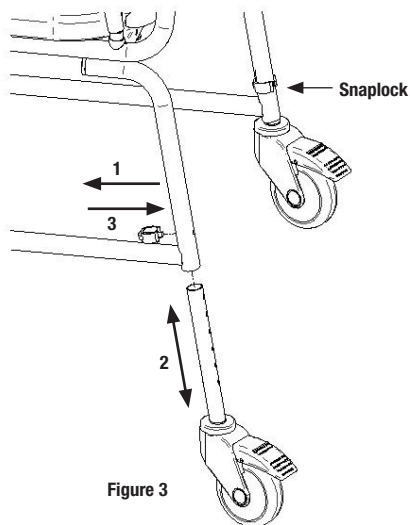
Figure 3

Adjust the rear wheel brakes by loosening the screw (4 on the drawing) and moving it either forwards or backwards to suit the height of the wheels. The brake pad is positioned approximately 1 cm/ 0.40 inches from the tire in the unlocked position. After adjustment, retighten the screw.

## Driving the chair

The user can independently drive the chair forward with the help of the large drive wheels. Grasp the outer ring that is mounted on the drive wheel, and push forward. The user can also operate the wheel brakes (one located on each side of the drive wheels) by pulling the handles backwards so the wheels are locked. When the brakes are pushed forward again the brakes are released.

## Maintenance and cleaning

To ensure longevity of the chair and safety for the user, the following should be included in the daily inspection:

- Inspect and clean the castors and wheels of dust and dirt
- Inspect and clean the wheel brakes of dust and dirt
- Check that all screws and nuts are properly tightened
- Check that the snaplock clips are securely fastened

After each use, or according to facility protocol, the Combi self-propelled chair may be cleaned/disinfected with non-abrasive, non-corrosive solutions. The PUR seat and backrest can be detached for cleaning, as can the bucket / bedpan holder (NOTE: the rear section of the bucket / bedpan holder must be lifted upwards before removing the two rods from the holes). The entire chair may also be washed / autoclaved at 85 degrees Celsius for 3 minutes. Avoid harsh solvents and chlorine. If alcohol is used to clean the black polyurethane, discolouration may occur. Ensure all parts are re-attached correctly prior to use.

## Reuse

Follow the above procedures. Defective or worn parts must be replaced.

## Warning

- Max. load Combi self-propelled (standard and wide model): 150 kg / 330 lbs
- Check the condition of the snaplocks frequently, and regularly check that the snaplocks are inserted correctly into the frame.
- Lock wheels when chair is stationary and also during transfers to and from the chair.
- The Combi self-propelled chair is not designed to be used as seating for extended-periods.
- The Combi self-propelled chair should always be used with the anti-tip feet. Regularly check the rubber tips for wear.
- Always use the Combi chair with armrests in place (folded down). Armrests (and footrests, if present) should only be folded up during transfer to and from the chair.
- If the castors and / or wheels are detached, it is important to mount them again correctly (see "Assembling the front castors & rear wheels" above). Ensure that all 4 legs (the castors and the anti-tip feet) are adjusted to the same height.
- Keep fingers away from the armrest joints to avoid pinching during raising and lowering.
- Use the chair only on a flat, even surface.
- Insert or remove the bedpan/bucket when the chair is not in use.
- Adjust height of chair only when not in use
- Do not store the chair in direct sunlight as the black surface may become hot and uncomfortable for the user.
- When using the Combi self-propelled, avoid wearing loose clothing that might become caught in the wheel brakes.
- Do not put your fingers in the drain holes of the back rest, or your toes in the drain holes of the foot rests, as there is a risk of them getting stuck.

## Disposal

The plastic parts should be sorted as incinerated and stainless steel parts as metal.

## List of configurations and accessories

### Combi chair

201921314	Combi	self-propelled, standard width
201930314	Combi	self-propelled, wide

### Accessories

200360004	Headrest	For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.
201120040	Seat with forward tilt	For all versions of Combi attendant operated and Combi self-propelled.
200382004	Hip belt	For all Combi chair models and versions.
201040314	Front cross bar	For all Combi chair models and versions with standard width. Recommended for use with the Armrest lock accessory.
201030310	Arm rest lock	1 pair. For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.
201070314	Lateral trunk support, right	For all Combi chair models and versions.
201072314	Lateral trunk support, left	For all Combi chair models and versions.
201060004	Amputee support	For all Combi chair models and versions.
201061004	Leg support	For Amputee support and all Combi chair models and versions.
200810004	Footrests	1 pair. For all Combi chair models and versions. Recommended for use with all models and versions of Combi chairs.
200384004	Heel and calf strap	For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.
853100604	Transport seat cover	For all Combi chair models and versions.
853100034	Bucket with lid	Keyhole shape, white plastic. For all Combi chair models and versions.
853100033	Bucket with lid	Round, grey plastic. For all Combi chair models and versions.


## Materials

Frame:	Stainless steel
Surface coating frame:	Powder coating
Seat, backrest, armrests:	Polyurethane (PUR)
Snaplock clips:	Chrome pin/Polypropylene

## Technical Specifications

Seat height interval:	50.5-55.5 cm / 19.9"-21.8", adjustable
Seat measurements, width x depth:	51 cm x 43.5 cm / 20" x 17.1"
Toilet opening, width x depth:	21 cm x 31 cm / 8.2" x 12.2"
Distance between armrests:	Standard: 43 cm / 17"
	Wide: 53 cm / 20.8"
Armrest, length:	48 cm / 18.8"
Armrest, width:	5 cm / 2"
Backrest, width x height:	41 cm x 31 cm / 16.1" x 12.2"
Outer measurements, width x depth:	Standard: 70 cm x 73 cm / 27.5" x 28.7"
	Wide: 80 cm x 73 cm / 31.4" x 28.7"
Castors:	Ø 100 mm / 4"
Self propelling wheels:	Ø 24"
Total weight:	Standard: 22 kg / 48 lbs
	Wide: 23 kg / 50 lbs
Max. user weight:	150 kg / 330 lbs

<b>Max load:</b>	201904314:
	150 kg / 330 lbs
	201903314:
	150 kg / 330 lbs
	201903314:
	150 kg / 330 lbs
	201903314:
	50 kg / 110 lbs

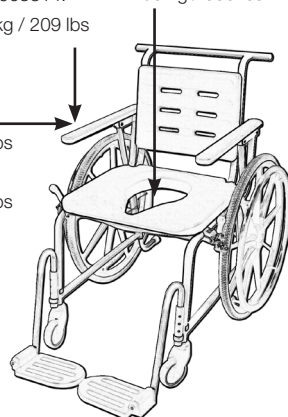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
<b>Max. belasting:</b>
201904314:
50 kg
201903314:
50 kg

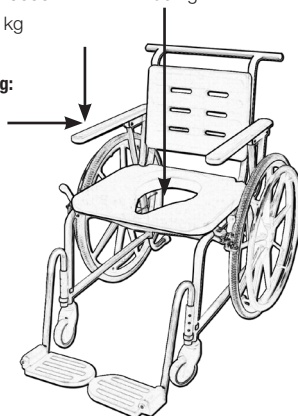## **Always make sure that you have the correct version of the manual**

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## **For questions about the product and its use**

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.*

Handicare AB  
Torshamnsgatan 35  
SE-164 40 Kista , SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)