User manual – English

Bruksanvisning – Svenska

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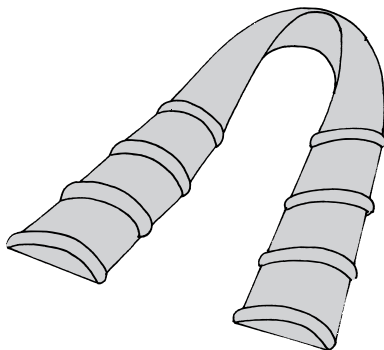
REF

6085

SWL: 160 kg/ 350 lbs

Table of contents

English	4-5
Svenska	6-7
Norsk	8-9
Dansk	10-11
Suomi	12-13
Deutsch	14-15
Nederlands	16-17
Français	18-19
Italiano	20-21
Español	22-23

SitWalk, 245 x 12cm/96 x 5", is an accessory to StandUP to facilitate the caregiver to assist a user on the seat to the StandUP. It is equipped with several handles to ensure a good working stance for the caregiver.

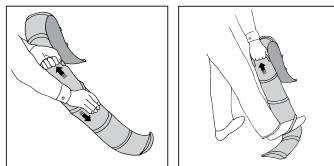
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

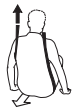
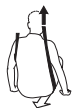
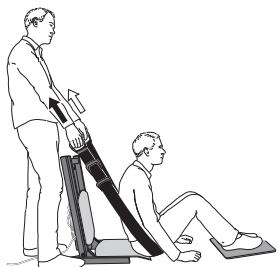
Placement and removal of SitWalk

Place SitWalk under the thighs and close to the seat, with the handles facing out from the body. Pull on the handles alternately from the right and left side, so that the strap slides against the thighs and inward towards the buttocks.

Raising the user's feet from the floor will relieve some of the weight of the thighs from the seat, thereby making repositioning easier.

Remove SitWalk by pulling alternately on the handles, so that it slides farther out on the thighs until it can be fully removed without moving the user.

Using SitWalk

Help the user to a seated position and place the user's feet on an AntiSlip. Then, place StandUp behind the user's back and help the user to move to the seat. This can be done in two ways:

1. Place the SitWalk under the user's seat and pull the user onto the StandUp seat. Ask the user to assist by applying counter pressure, if possible.
2. Place the SitWalk under the user's seat and help the user to "sit-walk" onto the StandUp seat. Ask the user to assist by applying counter pressure, if possible.

Material

Polyester

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com