



Max: 200kg/440 lbs

CalfStrap is an accessory for MiniLift125/MiniLift200. It is a padded strap that gives the patient extra support and a sense of security when used in combination with MiniLift125/MiniLift200. CalfStrap is fastened via clasps at the side of the leg support, and the length is easily adjusted to suit the patient. On MiniLift125/MiniLift200 models, a hole at the back of the knee support is used to mount the fastener for the CalfStrap.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection


Check the condition and function of the product regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.
Keep the manual where it is accessible to users of the product.
Do not leave the patient unattended during a transfer situation.

Placement and use

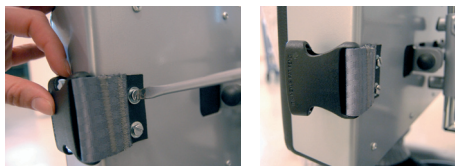

Screw the clasps for the CalfStrap to the back of the knee support on MiniLift125/MiniLift200. Use the accompanying spacers/washers between the leather and the screws.



Adjust the length of the strap to suit the patient, allowing sufficient support behind the calves. Then, place the CalfStrap behind the patient's calves and fasten it with the clasps, one on either side.

Care of the product:

Read the product label.



Do not use rinsing agent.
To prolong product life, avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Handicare AB
Veddestav. 15. Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8 -557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

