



SoftLegSupports are padded covers that can be applied to the leg sections of slings with separate leg supports. SoftLegSupports are used mainly for patients who are sensitive to pressure. They are sold in pairs and are easy for the caregiver to apply.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

Application of SoftLegSupport



1



2

1. The SoftLegSupport has a longer long side and a shorter long side, as well as a longer short side and a shorter short side.

2. Apply the SoftLegSupport so that the longer long side is placed towards the middle of the sling and the longer short side is towards the sling's back section. Apply on support to each leg section. Use the sling as you would normally. Read the manual for the sling model you intend to use.

Material:

Polyester: Durable material with low friction. Withstands high laundry temperatures.

Care of the product:

Read the product label.



Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

Size		XS/Orange	S/M/Red/Yellow	L/XL/Green/Purple
Rek.	lbs	40-80	70-190	180-450
Patient weight	kg	20-35	30-90	80-200

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Handicare AB

Veddestav. 15. Box 640 Tel: +46 (0)8 -557 62 200

SE-175 27 Järfälla Fax: +46 (0)8-557 62 299

SWEDEN

Internet: www.handicare.com

E-mail: info@handicare.se

