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## Manual - RFL X4

Manual nr: 00796 - EN, Nord Ver. 2 101209

*Ready for life is a series of assistive devices that are designed to give patients the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions of home-care and similar settings. Ready for life offers the right quality, the right function and the right performance at the right price. The transfer assistive products can be used either independently or as a complement to products in the SystemRoMedic range.*

Veddestav. 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299

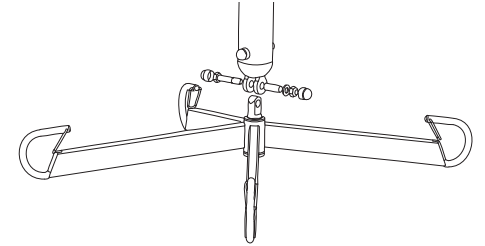
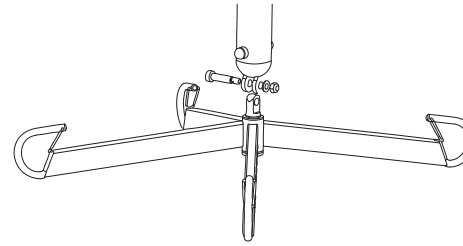
Internet: www.handicare.com  
E-mail: info@handicare.se

**Assembly**

**Montering**

**Asennus**

**Assembly**

**English**

**Montering**

**Norske**

**Asennus**

**Suomi**

**Installation of Sling bar RFL X4 on mobile and overhead lift units:**

The four-point sling crossbar comes with a locking nut for installation on lift units. Install using existing screw fasteners.

**NOTE!** Locking nuts must not be reused.

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## English

Slingbar RFL X4 is a four-point sling crossbar that is intended for use with Handicare-approved mobile and overhead lifts. It is designed for use with correctly fitted slings for lifting patients. Slingbar RFL X4 is designed to provide more space in the sling, for example, for obese and/or pain-sensitive patients. Slingbar RFL X4 is made of steel.

SWL: 300 kg/661 lbs

## Functional inspection

### Visual inspection

Inspect the sling bar regularly. Check to ensure that material is free from damage.

### Before use:

Ensure that the product is correctly assembled.  
Check sling bar connection and safety latch function.

## Always read the manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

The lift may only be used by persons who have received instruction in the operation of the lift.

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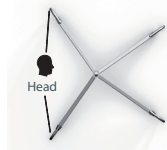
## Important information

- Lifting accessories must be properly trial fitted and tested in relation to the patient's needs and functional ability.
- Do not leave the patient unattended during a transfer situation.
- NOTE!** The lift sling's back loops are connected to the wider section of the slingbar (see image).

## Safe working load

Different products on the same lift system (lift unit, slingbar, sling, scales and other lifting accessories) may have different allowable safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

## Cleaning

Clean the lift with warm water or rubbing alcohol and ensure that the castor wheels are free of dirt and hair. Do not use cleaning agents containing phenol or chlorine, as this may damage the material.

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