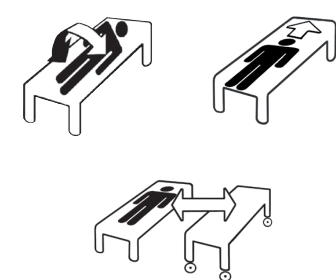
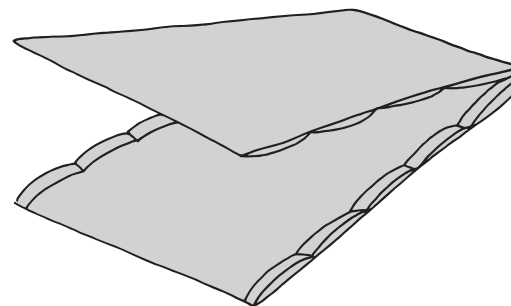
Manual - ReadySheet

Manual nr: 00839 - En, Nord Ver. 5 110211
Max 180kg/397lbs

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

English

Svenska

Norske

Danske

ReadySheet is a simple and versatile assistive device for application of lifting slings and for repositioning in one location, for example, for turning or moving the user higher up in bed.

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Functional inspection

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Kontrol af funktion

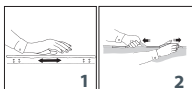
Visual inspection
Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

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

Mechanical load
Test the friction of the material by pressing it against the underlying surface and rolling it back and forth (1). Note whether it still glides as easily as it did. Compare with a new product if possible.

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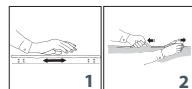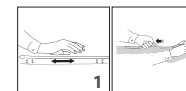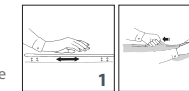
Pull on the handles to check seams and strength (2).

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Always read the instructions

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Les alltid bruksanvisningen

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Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

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Do not leave the patient unattended during a transfer situation.

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
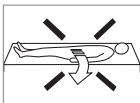
Note

Always remove the product after a transfer is completed. ReadySheet has a very low-friction surface. Where there is a risk of the patient sliding out of bed, use bed rails.

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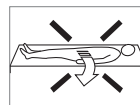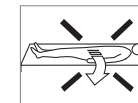
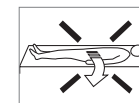
Placement/removal

Open

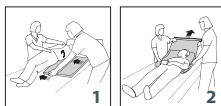
ReadySheet is placed under the patient with the handles facing downward. Lean the patient to one side, make use of the sheet, and place the folded product under the patient (1). Lean the patient in the opposite direction and pull the ReadySheet out on the other side.

Remove the ReadySheet by pulling it upward and out from underneath (2).

Tubular

The tubular version can be placed according to the above description, or it can be rolled up and then folded out under the patient (3). With the patient lying flat, place the roll under the knees. With one caregiver working on either side of the bed, unroll stepwise until the entire roll is fully rolled out under the patient.

Remove the ReadySheet by pulling it out from underneath at a slight upward angle.

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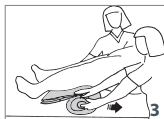
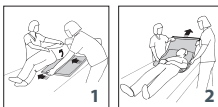
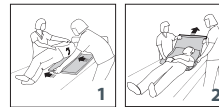
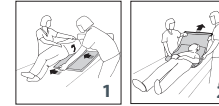
Turning

Open

For turning in bed, a ReadyLett is bedded around the mattress for even better glide.

Place the ReadySheet under the patient and push/pull the patient laterally (1) before turning (2).

English

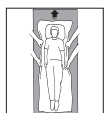
Higher up in bed

Open

For repositioning higher up in bed, a ReadyLett is bedded around the mattress. Place the ReadySheet under the patient and pull the patient higher up in bed.

Tubular

When repositioning a patient higher up in bed using a tubular ReadySheet, place the sheet under the patient as described earlier. Then, grasp the handles on either side and pull the patient higher up in bed.


English

Transfer between bed and gurney

Open

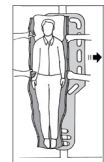For transfers between bed and gurney, use EasyGlide 170x59 as a bridge. Place the ReadySheet under the patient in bed and pull/push the patient over to the gurney. Remember to lock the wheels on both the bed and the gurney before starting the transfer.

Size:

11440	175 x 75
11430	195 x 75
11450	150 x 75
11445	175 x 75 Tubular

Material:

Polyamid, polypropylene

Care of the product:

Read the product label.

Do not use rinsing agent.
To prolong product life, avoid tumble-drying.

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