# Backrest

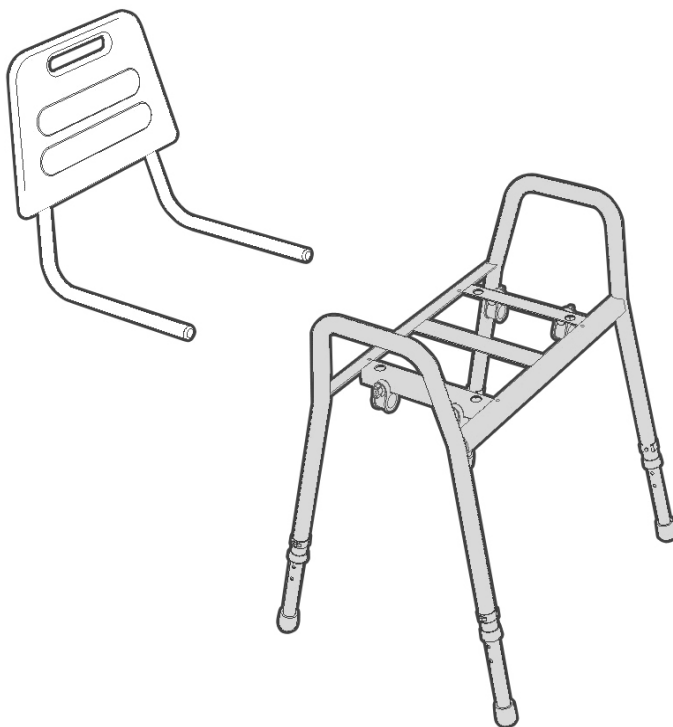
Manual - English

Manual - Svenska

Manual - Danske

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Art.no.: 200180312 / LI2002.0014-02

**MAX: 150 kg/330 lbs**

Manual no: LI09509.407 ver.3 140616

**EN Visual inspection**

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## EN - Please read the instructions before you start using this product

### Intended use

For optimal safety, Handicare's Shower stool soft seat (Art.no.: 200501312/200506312) and Shower stool plastic seat (Art.no.: 200500312) can be provided with a backrest.

The backrest is made of polyurethane, a material with good non-slip properties.

### Assembly – see drawing

In case your shower stool has no welded rings under the seat, please follow the instructions A. In case the rings are already welded to the shower stool, please follow the instructions B.

### Maintenance

To be wiped clean after use. Use domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied.

The backrest slats can be detached for cleaning. Polyurethane can tolerate temperatures up to 85 °C / 3 minutes.

The backrest frame is made of stainless steel.

### Warning!

- Max. load on stool with backrest: 150 kg.
- Only use the backrest as described in the paragraph "Intended use".
- When the backrest is mounted in position it is not necessary to place the stool against a wall.
- Check the correct position of the snaplocks before use.
- Check the condition of the snaplocks frequently. Regularly check that the snaplocks are correctly inserted into the tubing.

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**EN 1.** Follow these instructions for a stool without welded rings. Remove the seat from the frame.

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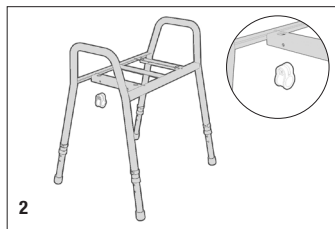
**EN 2.** Place the holder on the frame (4x). Make sure the holes are lined up.

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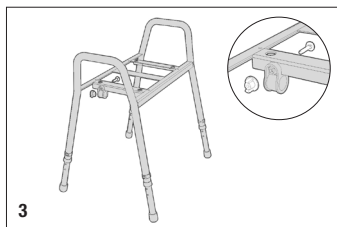
**EN 3.** Place the bolt through the frame and the holder, starting from the inside. Screw it together with the wingnut.

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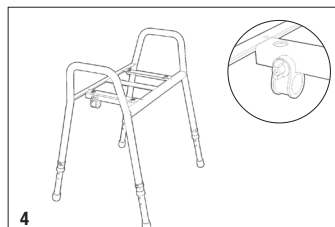
**EN 4.** Check if the wingnut is firmly tightened.

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**EN 5.** Mount the backrest by inserting the tubing through the holders.

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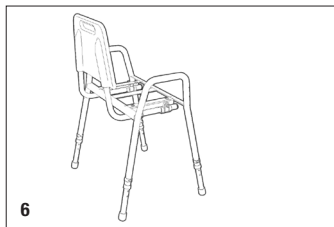
**EN 6.** Position the backrest to the correct seat depth. Attach the snaplocks, one on each side of the rings.

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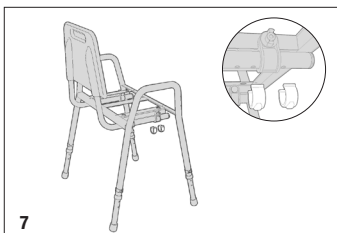
**EN 7.** Check if the clips are completely pressed against the tube.

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**EN 8.** Reattach the seat in its original position.

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**EN 1.** Follow these instructions for a stool with welded rings attached.

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**EN 2.** Mount the backrest by inserting the tubing through the holders.

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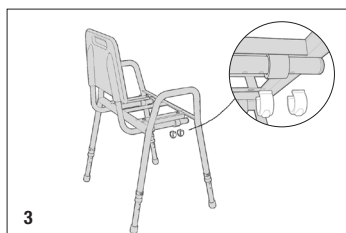
**EN 3.** Position the backrest to the correct seat depth. Attach the snaplocks, one on each side of the rings.

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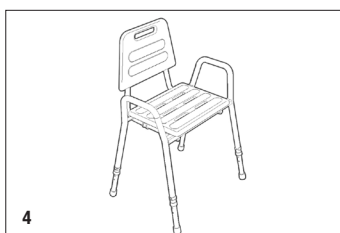
**EN 4.** Check if the clips are completely pressed against the tube.

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## Maintenance

**EN** Clean regularly with normal household detergent. Do not use aggressive cleaners. Handicare is not liable for damage caused by improper mounting/use.

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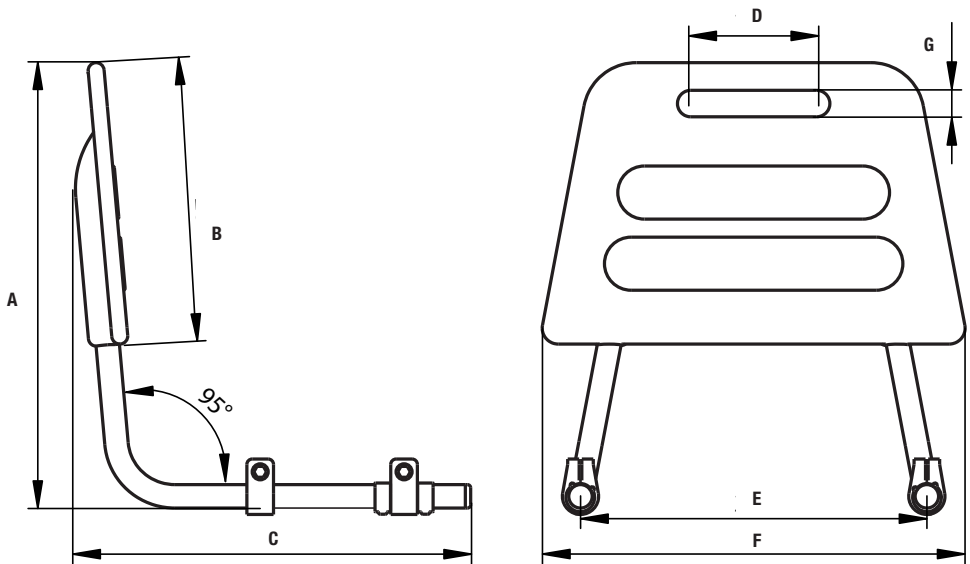
***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, Poland, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various power and manual wheelchairs, scooters, seating systems, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise- and recline chairs, and bathing and toileting products.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com)

Handicare AB  
Veddestav. 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)

Internet: [www.handicare.com](http://www.handicare.com)