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## **EN - Please read the instructions before using the product.**

### ***These instructions apply to the armrest lock for the Combi chair.***

#### **Intended use**

The armrest lock can be mounted as an accessory onto all Combi chairs, and insures that the armrests cannot inadvertently fold up, for example, in the Combi Manual Tilt Chair and the Combi Power Tilt Chair, when the chairs are tipped over backwards. The armrest lock can be used to advantage together with the front cross bar combi chair.

#### **Installation**

Loosen and remove the two front screws on the armrest. Mount the armrest lock on the armrest using the supplied longer screws (1) and the nylon rings (2). Place the two rings on both sides between the U-profile of the armrest and the armrest lock. The bend in the armrest lock must point backwards, as shown in the sketch.

#### **Adjustment and use**

Press down the armrest lock vertically under the armrest to insure against inadvertently swinging up of the armrest. Be sure that the armrest lock is pressed down completely to the frame for optimal use.

Pull the lock backwards, to lift up the armrest. The armrest can then be moved freely.

#### **Maintenance**

The armrest lock can be cleaned with ordinary cleaning products without polishing or etching properties. It is not recommended to use solvents or chlorine. Dry off with a clean cloth afterwards.

#### **Warning!**

- Must only be used as described under the section "Intended use".
- Should always be installed when the users are agitated or overly active.
- Check regularly that the screws are tightened.
- Make certain that the armrest lock is pressed completely down to the frame for optimal use.
- Be aware of the danger for being pinched when the armrest is being folded up or down.

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## **Always make sure that you have the correct version of the manual**

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## **For questions about the product and its use**

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.*

Handicare AB  
Torshamnsgatan 35  
SE-164 40 Kista , SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)