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EN - Please read the instructions before using the product.
These instructions apply to the Footrests Combi chair.

Purpose

The footrests can be mounted on all models of Combi chairs, giving support to the user's feet.

Assembly

Select a hole at the desired height and insert the snaplock clips (2) securely into the tubing. Slot the footrest tubing into the frame holder, letting it rest securely onto the snaplock pin. Lift the tubing slightly to swing out at 90°.

Adjustment

The footrest height can be adjusted by changing the position of the snaplock pins in the frame tubing. There are 4 positions with 25 mm / 0.9 " increments.

Maintenance

Clean with regular household cleaning agents with non-corrosive or non-abrasive properties. Do not use solvents or chlorine.

Caution!

- Use only as described under "Purpose".
- Do not stand on the footplates when the footrests are mounted onto the chair.
- Footrests should always be used when transporting the user in the chair.
- Footrests should always be used with tilt chairs.
- The footrests must be swung out at 90°, or the footplates flipped up, when the user is dismounting the chair.

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Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com