EasyGlide

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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REF

5045

SWL: 150 kg/330 lbs

Table of contents

English	4-5
Svenska	6-7
Norsk	8-9
Dansk	10-11
Suomi	12-13
Deutsch	14-15
Nederlands	16-17
Français	18-19
Italiano	20-21
Español	22-23

The new, smaller version of EasyGlide, oval are used in pairs to facilitate positioning further back in chair. Using EasyGlide, a more active user can push or shuffle himself backwards in the chair, into a correct and comfortable position.

The sliding boards are small and convenient, with soft, round shapes and smooth surfaces; features which make them very easy to place and remove and to always have readily available and close at hands.


Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Stability test

Carefully bend the EasyGlide to check strength and durability. The board should be both elastic and rigid. Discard the EasyGlide when its strength and durability are diminished.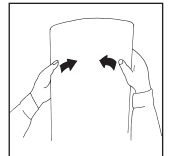

Always read the user manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Always make sure that you have the right version of the manual.


The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Important information

Do not leave the user unattended during a transfer situation.


Placement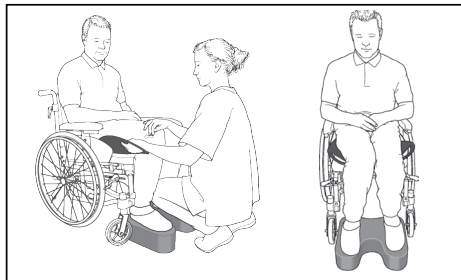


Place the EasyGlide by guiding it under the user's thighs. This is facilitated by placing a FootStool under the feet and thereby raising the user's thighs from the underlying surface. Guide the EasyGlide in under the user's seat.

Do the same on the other side.

Farther back in the chair

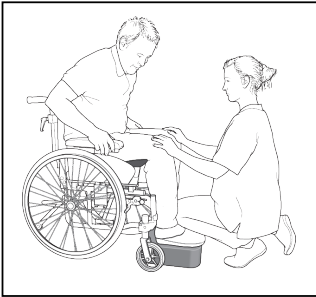
Ask the user to lean forward and push herself/himself farther back in the chair with the support of the armrests. The caregiver should remain in front of the user to ensure safety.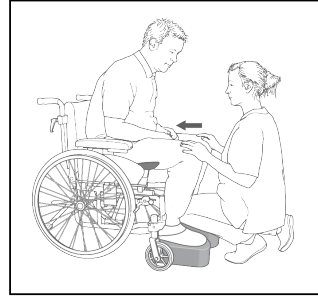

If the patient is unable to move farther back in the chair, the caregiver can push against the patient's knees to reposition the patient farther back in the chair. Ask the patient to lean forward, or ask another caregiver to assist.

Removal

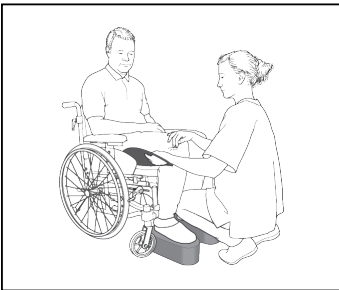The transfer slides are removed by grasping the handles and drawing outward. Prevent the user's body from moving by holding the user as the slides are pulled out.

Material

Polyethen

Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

CE Medical Device Class I

The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com