TopSheet

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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REF

1510

1515

1520

1511

SWL: 250 kg/550 lbs (1515, 1520)

SWL: 300 kg/660 lbs (1510, 1511)

Table of contents

English	4-6
Svenska	7-9
Norsk	10-12
Dansk	13-15
Suomi	16-18
Deutsch	19-21
Nederlands	22-24
Français	25-27
Italiano	28-30
Español	31-33
<hr/>	
Size	34
Combination	34
Accessories	34

TopSheet is used together with a ceiling or mobile lift to avoid heavy manual handling when turning a user.

TopSheet is used for incontinent, bedridden users who must be turned often. TopSheet is also an excellent aid when you need to hold a heavy user in a lateral position, for instance when assisting with personal hygiene or dressing wounds.

TopSheet is also very suitable when a nervous, demented user in palliative care needs turning, as the process is calm and gentle and the load is spread over the entire body area.

Check Safety

Visual inspection


Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.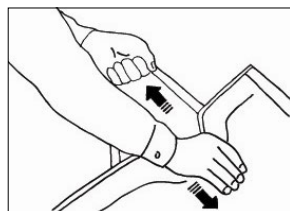

Always read the manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.


Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.


Important information


Do not leave the user unattended during a transfer situation.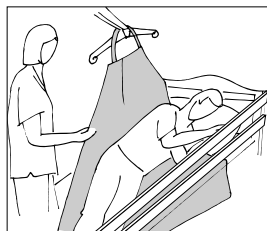

To ensure safety, a caregiver should stand on the side towards which the user is being turned; alternatively, bed rails can be used. Never leave the user unattended.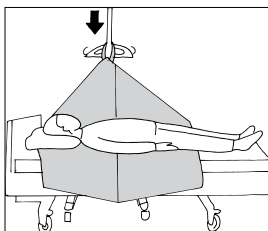

When a mobile lift is used, the slingbar should be placed directly over the suspension loop.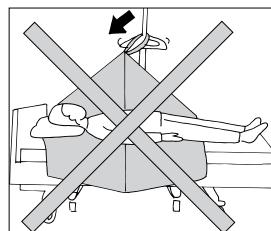

Lifting at a bias may entail risk for tipping. Ensure that the lift wheels can be moved laterally under the bed.


Placement


TopSheet 1520 is placed over TurnSheet 1580, 1590 or WendyLett4Way to minimize friction during turning. TopSheet 1510, which has a low-friction fabric on the underside, can be placed directly over the mattress cover or on WendyLett.

Read the manuals for other products used in combination with TopSheet.

Position the user so that his/her hips are resting on the bottom section of the TopSheet. This makes it easier to change the sheet and enables easier access for cleaning and washing.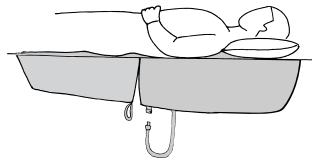



Turning the user with TopSheet

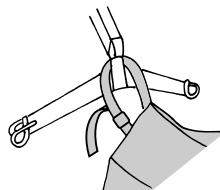The TopSheet suspension loop is hooked above the slingbar on a mobile or ceiling lift.


If bed rails are not used, a caregiver should stand beside the bed to ensure safety.

Start turn by raising the slingbar. The caregiver with the best view of the user operates the lift. Turn the user to the desired position.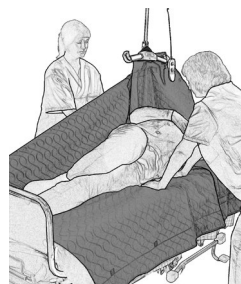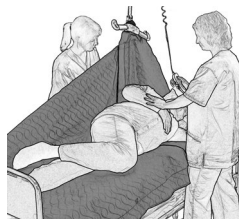


To ensure that the user is repositioned in the centre of the bed, some manual adjustment towards the centre of the bed may be necessary during turning.

Turning and hygiene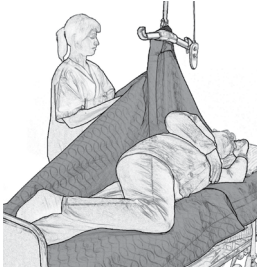

When the user has been turned to the desired position, the Velcro strip can be loosened behind the user.

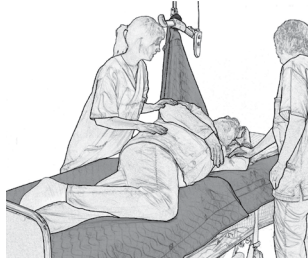Since the TopSheet now supports the user's back, the caregiver is free to attend to the user's hygiene.

Changing the bottom TopSheet section

Turn the user on one side (see instructions for turning with TopSheet). Loosen the Velcro strip on the section on which the user is lying and tuck it in as far as possible under the user. Take a new bottom section, fasten it along the Velcro strip, and tuck it under the user as far as possible.

Turn the user back on to his/her back, move the lift to the other side, and turn the user in the opposite direction.

Remove the bottom section that is to be replaced. Pull the Velcro strip out and fasten the new bottom section to it.

Material

Polyester, polyamid

Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Combination

Art no.	Combination
1510	Mattress cover 1593 or 1594 or WendyLett
1515	Mattress cover 1593 or 1594 or WendyLett
1520	TurnSheet and mattress cover 1590 or WendyLett + WendyLett4Way

Accessories

Art no.	
1509	1400 x 1600 mm/55.1 x 62.9 inch (Lower part to 1510)
1519	800 x 1400 mm/31.4 x 55.1 inch (Lower part to 1520)
1514	850 x 1400 mm/33.4 x 55.1 inch (Lower part to 1515)

Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com