TurnSheet

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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REF

1592 1594

1588 1595

1589 1587

1597 1580

1583 1590

1593 1586

SWL: 300 kg/660 lbs

Table of contents

English	4-6
Svenska	7-9
Norsk	10-12
Dansk	13-15
Suomi	16-18
Deutsch	19-21
Netherlands	22-24
Français	25-27
Italiano	28-30
Español	31-33
<hr/>	
Size and Accessories	34
Combination	34

For heavier users and users who need assistance with turning in bed.

Being confined to a lying position for a long period is a strain on the body and soul. It gets even worse if pressure sores develop. With the help of a frequent turning schedule and TurnSheet, life becomes much easier both for the user and for the caregiver.

TurnSheet is made of microfiber and Handicare's own sophisticated sliding material ULF (Ultra Low Friction) with a thin padding of polyester.

TurnSheet lies on top of Mattress cover with smooth, low friction material in the centre and rough anti-slip surfaces at the sides. This minimizes the risk of the user sliding out of bed.

Check Safety


Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

High friction - Ensure that the surface is rough and prevents the transfer aid from moving. Compare with a new product.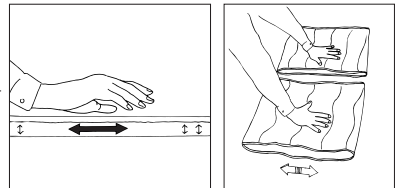

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of manuals are available for downloading from our website, www.handicare.com.


Important information



Be aware that there is a risk of the user sliding out of bed. Take care to ensure that the mattress is sufficiently wide, so that there is anti-slip material on both edges of the mattress cover. Do not leave the user unattended without first raising the bed rails. Tuck the Slide panel under the mattress.

Placement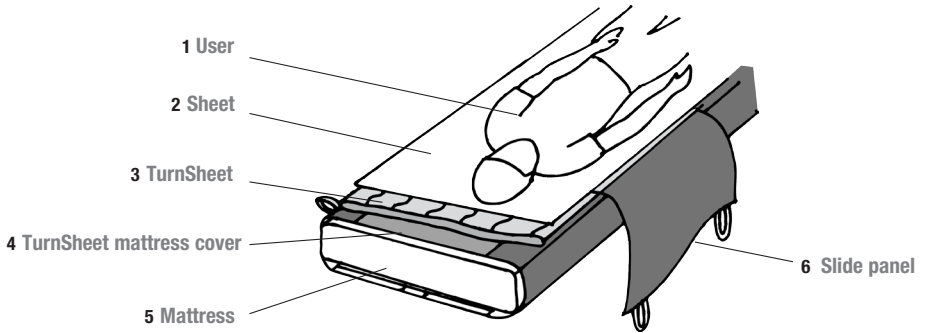

TurnSheet is placed and bedded in the bed in the order described above.
 The mattress cover is secured around the mattress with Velcro strips.
 The turning cover is placed on top, with the sliding surfaces facing each other.



Turn manually with TurnSheet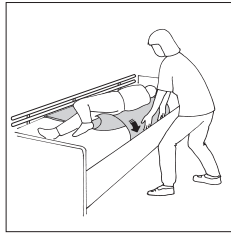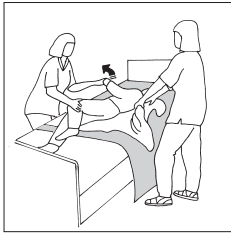

Pull the user to one side of the bed. Smooth out the sheet and turning cover. Pull on the sheet and turn the user to the desired position.

Tuck the sheet under both sides of the mattress and raise the bed rails.

Turning with TurnSheet, two parts (Art no. 1597)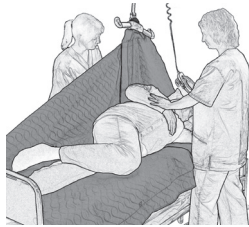
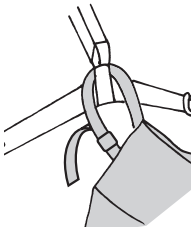


The TurnSheet's suspension loop is hooked above the slingbar on a mobile or overhead lift.

If bed rails are not used, a caregiver should stand beside the bed to ensure safety.

Start turn by raising the slingbar. The caregiver with the best view of the user operates the lift.

Turn the user to the desired position.

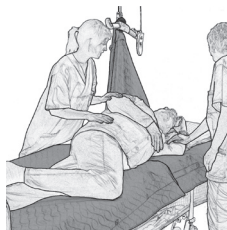
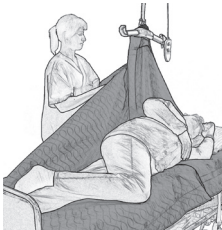


To ensure that the user is repositioned in the centre of the bed, some manual adjustment towards the centre of the bed may be necessary during turning.

When the user has been turned to the desired position, the Velcro strip can be loosened behind the user.

Since the TurnSheet now supports the user's back, the caregiver is free to attend to the user's hygiene.

Using slide panels

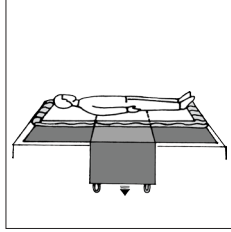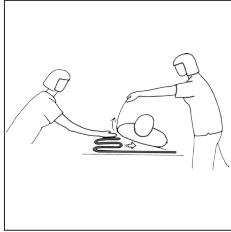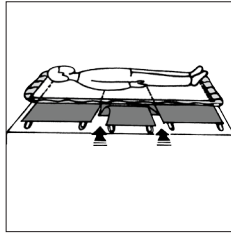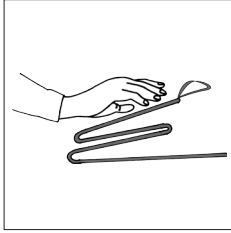Mattress cover 140cm/ 55" has two slide panels and mattress cover 200 cm/78" has three.

There is friction material under the slide panels.

The slide panels can be removed to enable positioning of the user in bed. For example, the slide panel around the hips can be removed to prevent the user from sliding downward when he/she attempts to sit up in bed.

Placing and removing the slide panels is easy, since the slide panel is folded against itself and slides in under the user. Press down and pull forward. This is easier if one caregiver can tilt the user somewhat.

When removing the Slide panel, tuck it under the mattress to prevent it from hanging down on the floor.

Material

Polyamid, polyester, microfibre

Care of the product

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

handicare

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com