# TurnSheet

SystemRoMedic™

User manual - English

Bruksanvisning - Svenska

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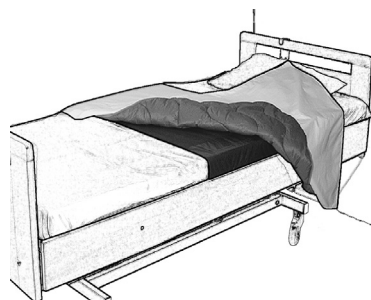
**REF**

2060

2080

2081

**SWL: 300 kg/660 lbs**

## Table of contents

<b>English</b> .....	4-5
<b>Svenska</b> .....	6-7
<b>Norsk</b> .....	8-9
<b>Dansk</b> .....	10-11
<b>Suomi</b> .....	12-13
<b>Deutsch</b> .....	14-15
<b>Netherlands</b> .....	16-17
<b>Français</b> .....	18-19
<b>Italiano</b> .....	20-21
<b>Español</b> .....	22-23
<hr/>	
<b>Combination</b> .....	24
<b>Size and Accessories</b> .....	24

Enables a more active user to turn and change position in bed. TurnSheet (65 x 130) has been developed for users with back problems or reduced functional abilities who need help with moving in bed.

TurnSheet is made of microfiber and Handicare's own sophisticated sliding material ULF (Ultra Low Friction) with a thin padding of polyester.

TurnSheet lies on top of Mattress cover (item number 2081) with smooth, low friction material in the centre and rough anti-slip surfaces at the sides. This minimizes the risk of the user sliding out of bed.

## Check Safety

---

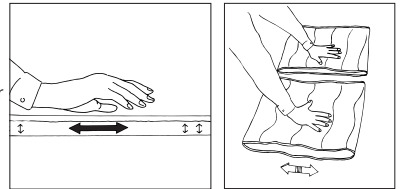
### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

High friction - Ensure that the surface is rough and prevents the transfer aid from moving. Compare with a new product.

## Always read the user manual

---

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

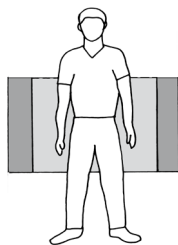
## Important information

---


Be aware that there is a risk of the user sliding out of bed. Take care to ensure that the mattress is sufficiently wide, so that there is anti-slip material on both edges of the mattress cover.

## Placement

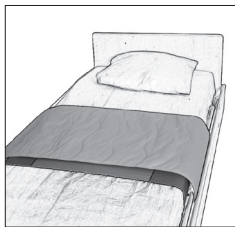
The mattress cover is placed with the sliding surface in the middle of the bed and the rough friction surfaces evenly distributed on both sides. The mattress cover is secured around the mattress with Velcro strips.


The turning cover is placed over the mattress cover. A draw sheet can be placed on top of the turning cover.

## Using TurnSheet

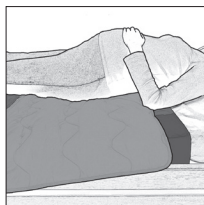The user can use TurnSheet himself/herself if he/she is able to turn with a little help from a caregiver.

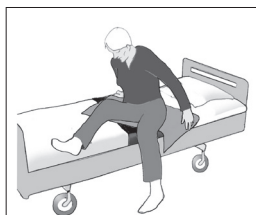Since the TurnSheet glides easily, it makes turning easier for the caregiver. Remember to work in walking stance with knees bent, arms straight, and shift the weight.

TurnSheet can also be used to help the user to rise to a seated position at the edge of the bed. The user can rotate easily to one side and lift the feet over the edge of the bed.

## Material

Polyamid, polyester, microfibre

## Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

**CE** Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)